

GE Healthcare

LCC Magnet Monitor Hardware Installation

Technical Publication



Direction 2230681
Rev 8
© 2005, General Electric Company,
All Rights Reserved

TABLE OF CONTENTS

LANGUAGE POLICY	4
1 - EQUIPMENT SPECIFICATIONS	11
1-1 Equipment Specifications for Granite Magnet Monitor	11
1-2 Equipment Specifications for Advantech Magnet Monitor	12
2 - INTRODUCTION	15
2-1 Overview	15
2-2 System Requirements	15
2-3 Document Application.....	15
2-3-1 Core Magnet with Sumitomo Compressor.....	15
2-3-2 S-Series Magnets through CX.....	15
3 - TOOLS AND EQUIPMENT	16
3-1 Required Tools/Equipment.....	16
3-2 Optional Parts	16
3-3 Magnet Monitor Description.....	17
4 - INSTALLATION PROCEDURES	17
4-1 Magnet Room Attachments	17
4-1-1 Before Beginning	17
4-1-2 Penetration Panel Connection	18
4-1-3 Connecting Magnet Room Cables (All Systems)	19
4-1-4 Connecting Reed Switch	21
4-1-5 Attaching Preamp to Magnet	22
4-2 Installing Magnet Monitor (All Systems)	22
4-2-1 Physical Installation	22
4-2-2 Changing Operating Voltage of Magnet Monitor	24
4-2-3 Configuring Magnet Monitor for External Modem (Granite Unit Only)	25
4-2-4 Installing External Modem (Advantech Unit Only).....	25
4-3 Sumitomo Shield Cooler Sensor Installation (Core and Twin with TGWC Only).....	26
4-3-1 Before Beginning	26
4-3-2 Attaching Water Flow and Temperature Sensor	26
4-4 Installing Magnet Monitor UPS (Optional).....	28
4-5 Magnet Monitor Calibration and Final Installation	29
4-5-1 Calibrating Magnet Monitor Helium Level Meter	29
4-5-2 Connecting Magnet Monitor to System Cabinet.....	32
4-6 Installing Global Modem (Advantech Magnet Monitor Only)	34
4-7 Installing MUX Box (Optional).....	34
4-8 Installing Remote Alarm (Optional for Advantech Only).....	34
5 - SOFTWARE CONFIGURATION	35
6 - FUNCTIONAL CHECKS	37
6-1 Magnet Monitor	37
6-2 Remote Alarm Check.....	38
6-2-1 Functionality of Remote Alarm Box	38
7 - Maintenance and Replacement.....	38
7-1 Maintenance	38
7-2 Magnet Monitor Replacement	38
8 - SCHEMATICS AND INTERCONNECTS	39
8-1 System Interconnect Diagram for Core Product with Water Flow/Temp Sensor	39
8-2 System Interconnect Diagram for S-Series and CX Magnets	40
8-3 System Interconnect Diagram for UPS (Option for Twin)	41
8-4 Magnet Monitor Internal Interconnects	42
9 - GLOSSARY	46
APPENDIX A – CONNECTING REMOTE ALARM BOX TO HOSPITAL	47
APPENDIX B – CONNECTING PRESSURE TRANSDUCER TO MAGNET	48
B-1 Connecting Pressure Transducer on S-II, S-III, S-III Magnashield and 0.5T MAX Magnets	48
B-2 Connecting Pressure Transducer on S-IV, S-V, S-X, and S-XC Magnets.....	49
B-3 Connecting Pressure Transducer on CX Magnets.....	51
REVISION HISTORY	53

DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent, the notation "Damage in Shipment" should be written on all copies of the freight or express bill before delivery is accepted or signed for by a General Electric representative or a hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14-day period.

To file a report:

- Call 1-800-548-3366 and select Option 6.
- Fill out a report at: <http://egems.med.ge.com/edq/home.jsp>.
- Contact your local service coordinator for more information on this process.

LANGUAGE POLICY

ПРЕДУПРЕЖДЕНИЕ

(BG)

ТОВА УПЪТВАНЕ ЗА РАБОТА Е НАЛИЧНО САМО НА АНГЛИЙСКИ ЕЗИК.

- АКО ДОСТАВЧИКЪТ НА УСЛУГАТА НА КЛИЕНТА ИЗИСКА ЕЗИК, РАЗЛИЧЕН ОТ АНГЛИЙСКИ, ЗАДЪЛЖЕНИЕ НА КЛИЕНТА Е ДА ОСИГУРИ ПРЕВОД.
- НЕ ИЗПОЛЗВАЙТЕ ОБОРУДВАНЕТО ПРЕДИ ДА СТЕ СЕ КОНСУЛТИРАЛИ И РАЗБРАЛИ УПЪТВАНЕТО ЗА РАБОТА.
- НЕСПАЗВАНЕТО НА ТОВА ПРЕДУПРЕЖДЕНИЕ МОЖЕ ДА ДОВЕДЕ ДО НАРАНЯВАНЕ НА ДОСТАВЧИКА НА УСЛУГАТА, ОПЕРАТОРА ИЛИ ПАЦИЕНТ В РЕЗУЛТАТ НА ТОКОВ УДАР ИЛИ МЕХАНИЧНА ИЛИ ДРУГА ОПАСНОСТ.

警告

(ZH-CN)

本维修手册仅提供英文版本。

- 如果维修服务提供商需要非英文版本，客户需自行提供翻译服务。
- 未详细阅读和完全理解本维修手册之前，不得进行维修。
- 忽略本警告可能对维修人员，操作员或患者造成触电、机械伤害或其他形式的伤害。

警告

(ZN-HK)

本服務手冊僅提供英文版本。

- 倘若客戶的服務供應商需要英文以外之服務手冊，客戶有責任提供翻譯服務。
- 除非已參閱本服務手冊及明白其內容，否則切勿嘗試維修設備。
- 不遵從本警告或會令服務供應商、網絡供應商或病人受到觸電、機械性或其他危險。

警告

(ZH-TW)

本維修手冊僅有英文版。

- 若客戶的維修廠商需要英文版以外的語言，應由客戶自行提供翻譯服務。
- 請勿試圖維修本設備，除非您已查閱並瞭解本維修手冊。
- 若未留意本警告，可能導致維修廠商、操作員或病患因觸電、機械或其他危險而受傷。

UPOZORENJE

(HR)

OVAJ SERVISNI PRIRUČNIK DOSTUPAN JE NA ENGLJESKOM JEZIKU.

- AKO DAVATELJ USLUGE KLIJENTA TREBA NEKI DRUGI JEZIK, KLIJENT JE DUŽAN OSIGURATI PRIJEVOD.
- NE POKUŠAVAJTE SERVISIRATI OPREMU AKO NISTE U POTPUNOSTI PROČITALI I RAZUMJELI OVAJ SERVISNI PRIRUČNIK.
- ZANEMARITE LI OVO UPOZORENJE, MOŽE DOĆI DO OZLJEDE DAVATELJA USLUGE, OPERATERA ILI PACIJENTA USLIJED STRUJNOG UDARA, MEHANIČKIH ILI DRUGIH RIZIKA.

VÝSTRAHA

(CS)

TENTO PROVOZNÍ NÁVOD EXISTUJE POUZE V ANGLICKÉM JAZYCE.

- PŘÍPADĚ, ŽE EXTERNÍ SLUŽBA ZÁKAZNÍKŮM POTŘEBUJE NÁVOD V JINÉM JAZYCE, JE ZAJIŠTĚNÍ PŘEKladU DO ODPOVÍDAJÍCÍHO JAZYKA ÚKOLEM ZÁKAZNÍKA.
- NESNAŽTE SE O ÚDRŽBU TOHOTO ZAŘÍZENÍ, ANIŽ BYSTE SI PŘEČETLI TENTO PROVOZNÍ NÁVOD A POCHOPILI JEHO OBSAH.
- PŘÍPADĚ NEDODRŽOVÁNÍ TĚTO VÝSTRAHY MŮŽE DOJÍT K PORANĚNÍ PRACOVNÍKA PRODEJNÍHO SERVISU, OBSLUŽNÉHO PERSONÁLU NEBO PACIENTŮ VlivEM ELEKTRICKÉHO PROUDU, RESPEKTIVE VlivEM MECHANICKÝCH ČI JINÝCH RIZIK.

ADVARSEL

(DA)

DENNE SERVICE MANUAL FINDES KUN PÅ ENGELSK.

- HVIS EN KUNDES TEKNIKER HAR BRUG FOR ET ANDET SPROG END ENGELSK, ER DET KUNDENS ANSVAR AT SØRGE FOR OVERSÆTTELSE.
- FORSØG IKKE AT SERVICERE UDSTYRET MEDMINDRE DENNE SERVICE MANUAL HAR VÆRET KONSULTERET OG ER FORSTÅET.
- MANGLENDE OVERHOLDELSE AF DENNE ADVARSEL KAN MEDFØRE SKADE PÅ GRUND AF ELEKTRISK, MEKANISK ELLER ANDEN FARE FOR TEKNIKEREN, OPERATØREN ELLER PATIENTEN.

WAARSCHUWING

(NL)

DEZE ONDERHOUDSHANDLEIDING IS ENKEL IN HET ENGELS VERKRIJGBAAR.

- ALS HET ONDERHOUDSPERSONEEL EEN ANDERE TAAL VEREIST, DAN IS DE KLANT VERANTWOORDELIJK VOOR DE VERTALING ERVAN.
- PROBEER DE APPARATUUR NIET TE ONDERHOUDEN VOORDAT DEZE ONDERHOUDSHANDLEIDING WERD GERAADPLEEGD EN BEGREPEN IS.
- INDIEN DEZE WAARSCHUWING NIET WORDT OPGEVOLGD, ZOU HET ONDERHOUDSPERSONEEL, DE OPERATOR OF EEN PATIËNT GEWOND KUNNEN RAKEN ALS GEVOLG VAN EEN ELEKTRISCHE SCHOK, MECHANISCHE OF ANDERE GEVAREN.

ENGLISH

(EN)

THIS SERVICE MANUAL IS AVAILABLE IN ENGLISH ONLY.

- IF A CUSTOMER'S SERVICE PROVIDER REQUIRES A LANGUAGE OTHER THAN ENGLISH, IT IS THE CUSTOMER'S RESPONSIBILITY TO PROVIDE TRANSLATION SERVICES.
- DO NOT ATTEMPT TO SERVICE THE EQUIPMENT UNLESS THIS SERVICE MANUAL HAS BEEN CONSULTED AND IS UNDERSTOOD.
- FAILURE TO HEED THIS WARNING MAY RESULT IN INJURY TO THE SERVICE PROVIDER, OPERATOR OR PATIENT FROM ELECTRIC SHOCK, MECHANICAL OR OTHER HAZARDS.

HOIATUS

(ET)

SEE TEENINDUSJUHEND ON SAADAVAL AINULT INGLISE KEELES.

- KUI KLIENDITEENINDUSE OSUTAJA NÕUAB JUHENDIT INGLISE KEELEST ERINEVAS KEELES, VASTUTAB KLIENT TÕLKETEENUSE OSUTAMISE EEST.
- ÄRGE ÜRITAGE SEADMEID TEENINDADA ENNE EELNEVALT KÄESOLEVA TEENINDUSJUHENDIGA TUTVUMIST JA SELLEST ARU SAAMIST.
- KÄESOLEVA HOIATUSE EIRAMINE VÕIB PÕHJUSTADA TEENUSEOSUTAJA, OPERAATORI VÕI PATSIENDI VIGASTAMIST ELEKTRILÖÖGI, MEHAANILISE VÕI MUU OHU TAGAJÄRJEL.

VAROITUS

(FI)

TÄMÄ HUOLTO-OHJE ON SAATAVILLA VAIN ENGLANNIKSI.

- JOS ASIAKKAAN HUOLTOHENKILÖSTÖ VAATII MUUTA KUIN ENGLANNINKIELISTÄ MATERIAALIA, TARVITTAVAN KÄÄNNÖKSEN HANKKIMINEN ON ASIAKKAAN VASTUULLA.
- ÄLÄ YRITÄ KORJATA LAITTEISTOA ENNEN KUIN OLET VARMASTI LUKENUT JA YMMÄRTÄNYT TÄMÄN HUOLTO-OHJEEN.
- MIKÄLI TÄTÄ VAROITUSTA EI NOUDATETA, SEURAUKSENA VOI OLLA HUOLTOHENKILÖSTÖN, LAITTEISTON KÄYTTÄJÄN TAI POTILAAN VAHINGOITTUMINEN SÄHKÖISKUN, MEKAANISEN VIAN TAI MUUN VAARATILANTEEN VUOKSI.

- ATTENTION**
(FR)
- CE MANUEL DE SERVICE N'EST DISPONIBLE QU'EN ANGLAIS.
- SI LE TECHNICIEN DU CLIENT A BESOIN DE CE MANUEL DANS UNE AUTRE LANGUE QUE L'ANGLAIS, C'EST AU CLIENT QU'IL INCOMBE DE LE FAIRE TRADUIRE.
 - NE PAS TENTER D'INTERVENIR SUR LES EQUIPEMENTS TANT QUE LE MANUEL SERVICE N'A PAS ETE CONSULTE ET COMPRIS
 - LE NON-RESPECT DE CET AVERTISSEMENT PEUT ENTRAÎNER CHEZ LE TECHNICIEN, L'OPÉRATEUR OU LE PATIENT DES BLESSURES DUES À DES DANGERS ÉLECTRIQUES, MÉCANIQUES OU AUTRES.
- WARNUNG**
(DE)
- DIESE SERVICEANLEITUNG EXISTIERT NUR IN ENGLISCHER SPRACHE.
- FALLS EIN FREMDER KUNDENDIENST EINE ANDERE SPRACHE BENÖTIGT, IST ES AUFGABE DES KUNDEN FÜR EINE ENTSPRECHENDE ÜBERSETZUNG ZU SORGEN.
 - VERSUCHEN SIE NICHT DIESE ANLAGE ZU WARTEN, OHNE DIESE SERVICEANLEITUNG GELESEN UND VERSTANDEN ZU HABEN.
 - WIRD DIESE WARNUNG NICHT BEACHTET, SO KANN ES ZU VERLETZUNGEN DES KUNDENDIENSTTECHNIKERS, DES BEDIENERS ODER DES PATIENTEN DURCH STROMSCHLÄGE, MECHANISCHE ODER SONSTIGE GEFAHREN KOMMEN.
- ΠΡΟΕΙΔΟΠΟΙΗΣΗ**
(EL)
- ΤΟ ΠΑΡΟΝ ΕΓΧΕΙΡΙΔΙΟ ΣΕΡΒΙΣ ΔΙΑΤΙΘΕΤΑΙ ΣΤΑ ΑΓΓΛΙΚΑ ΜΟΝΟ.
- ΕΑΝ ΤΟ ΑΤΟΜΟ ΠΑΡΟΧΗΣ ΣΕΡΒΙΣ ΕΝΟΣ ΠΕΛΑΤΗ ΑΠΑΙΤΕΙ ΤΟ ΠΑΡΟΝ ΕΓΧΕΙΡΙΔΙΟ ΣΕ ΓΛΩΣΣΑ ΕΚΤΟΣ ΤΩΝ ΑΓΓΛΙΚΩΝ, ΑΠΟΤΕΛΕΙ ΕΥΘΥΝΗ ΤΟΥ ΠΕΛΑΤΗ ΝΑ ΠΑΡΕΧΕΙ ΥΠΗΡΕΣΙΕΣ ΜΕΤΑΦΡΑΣΗΣ.
 - ΜΗΝ ΕΠΙΧΕΙΡΗΣΕΤΕ ΤΗΝ ΕΚΤΕΛΕΣΗ ΕΡΓΑΣΙΩΝ ΣΕΡΒΙΣ ΣΤΟΝ ΕΞΟΠΛΙΣΜΟ ΕΚΤΟΣ ΕΑΝ ΕΧΕΤΕ ΣΥΜΒΟΥΛΕΥΤΕΙ ΚΑΙ ΕΧΕΤΕ ΚΑΤΑΝΟΗΣΕΙ ΤΟ ΠΑΡΟΝ ΕΓΧΕΙΡΙΔΙΟ ΣΕΡΒΙΣ.
 - ΕΑΝ ΔΕ ΛΑΒΕΤΕ ΥΠΟΨΗ ΤΗΝ ΠΡΟΕΙΔΟΠΟΙΗΣΗ ΑΥΤΗ, ΕΝΔΕΧΕΤΑΙ ΝΑ ΠΡΟΚΛΗΘΕΙ ΤΡΑΥΜΑΤΙΣΜΟΣ ΣΤΟ ΑΤΟΜΟ ΠΑΡΟΧΗΣ ΣΕΡΒΙΣ, ΣΤΟ ΧΕΙΡΙΣΤΗ Ή ΣΤΟΝ ΑΣΘΕΝΗ ΑΠΟ ΗΛΕΚΤΡΟΠΛΗΞΙΑ, ΜΗΧΑΝΙΚΟΥΣ Ή ΑΛΛΟΥΣ ΚΙΝΔΥΝΟΥΣ.
- FIGYELMEZTETÉS**
(HU)
- EZEN KARBANTARTÁSI KÉZIKÖNYV KIZÁRÓLAG ANGOL NYELVEN ÉRHEŐ EL.
- HA A VEVŐ SZOLGÁLTATÓJA ANGOLTÓL ELTÉRŐ NYELVRE TART IGÉNYT, AKKOR A VEVŐ FELELŐSSÉGE A FORDÍTÁS ELKÉSZÍTTETÉSE.
 - NE PRÓBÁLJA ELKEZDENI HASZNÁLNI A BERENDEZÉST, AMÍG A KARBANTARTÁSI KÉZIKÖNYVBEN LEÍRTAKAT NEM ÉRTELMEZTÉK.
 - EZEN FIGYELMEZTETÉS FIGYELMEN KÍVÜL HAGYÁSA A SZOLGÁLTATÓ, MŰKÖDTETŐ VAGY A BETEG ÁRAMÚTÉS, MECHANIKAI VAGY EGYÉB VESZÉLYHELYZET MIATTI SÉRÜLÉSÉT EREDMÉNYEZHETI.
- AÐVÖRUN**
(IS)
- ÞESSI ÞJÓNUSTUHANDBÓK ER EINGÖNGU FÁANLEG Á ENSKU.
- EF AÐ ÞJÓNUSTUVEITANDI VIÐSKIPTAMANNS ÞARFNAST ANNAS TUNGUMÁLS EN ENSKU, ER ÞAÐ SKYLDA VIÐSKIPTAMANNS AÐ SKAFFA TUNGUMÁL AÞJÓNUSTU.
 - REYNIÐ EKKI AÐ AFGREIÐA TÆKIÐ NEMA AÐ ÞESSI ÞJÓNUSTUHANDBÓK HEFUR VERIÐ SKOÐUÐ OG SKILIN.
 - BROT Á SINNA ÞESSARI AÐVÖRUN GETUR LEITT TIL MEIÐSLA Á ÞJÓNUSTUVEITANDA, STJÓRNANDA EÐA SJÚKLINGS FRÁ RAFLOSTI, VÉLRÆNU EÐA ÖÐRUM ÁHÆTTUM.

AVVERTENZA
(IT)

IL PRESENTE MANUALE DI MANUTENZIONE E DISPONIBILE SOLTANTO IN INGLESE.

- SE UN ADDETTO ALLA MANUTENZIONE ESTERNO ALLA GEMS RICHIEDE IL MANUALE IN UNA LINGUA DIVERSA, IL CLIENTE E TENUTO A PROVVEDERE DIRETTAMENTE ALLA TRADUZIONE.
- SI PROCEDA ALLA MANUTENZIONE DELL'APPARECCHIATURA SOLO DOPO AVER CONSULTATO IL PRESENTE MANUALE ED AVERNE COMPRESO IL CONTENUTO
- IL NON RISPETTO DELLA PRESENTE AVVERTENZA POTREBBE FAR COMPIERE OPERAZIONI DA CUI DERIVINO LESIONI ALL'ADDETTO ALLA MANUTENZIONE, ALL'UTILIZZATORE ED AL PAZIENTE PER FOLGORAZIONE ELETTRICA, PER URTI MECCANICI OD ALTRI RISCHI.

警告
(JA)

このサービスマニュアルには英語版しかありません。

- サービスを担当される業者が英語以外の言語を要求される場合、翻訳作業はその業者の責任で行うものとさせていただきます。
- このサービスマニュアルを熟読し理解せずに、装置のサービスを行わないでください。
- この警告に従わない場合、サービスを担当される方、操作員あるいは患者さんが、感電や機械的又はその他の危険により負傷する可能性があります。

경고
(KO)

본 서비스 매뉴얼은 영어로만 이용하실 수 있습니다.

- 고객의 서비스 제공자가 영어 이외의 언어를 요구할 경우, 번역 서비스를 제공하는 것은 고객의 책임입니다.
- 본 서비스 지침서를 참고했고 이해하지 않는 한은 해당 장비를 수리하려고 시도하지 마십시오.
- 이 경고에 유의하지 않으면 전기 쇼크, 기계상의 혹은 다른 위험으로부터 서비스 제공자, 운영자 혹은 환자에게 위해를 가할 수 있습니다.

BRĪDINĀJUMS
(LV)

ŠĪ APKALPES ROKASGRĀMATA IR PIEEJAMA TIKAI ANGLŪ VALODĀ.

- JA KLIENTA APKALPES SNIEDZĒJAM NEPIECIEŠAMA INFORMĀCIJA CITĀ VALODĀ, NEVIS ANGLŪ, KLIENTA PIENĀKUMS IR NODROŠINĀT TULKĒŠANU.
- NEVEICIET APRĪKOJUMA APKALPI BEZ APKALPES ROKASGRĀMATAS IZLASĪŠANAS UN SAPRAŠANAS.
- ŠĪ BRĪDINĀJUMA NEIEVĒROŠANA VAR RADĪT ELEKTRISKĀS STRĀVAS TRIECIENA, MEHĀNISKU VAI CITU RISKU IZRAISĪTU TRAUMU APKALPES SNIEDZĒJAM, OPERATORAM VAI PACIENTAM.

ISPĒJIMAS
(LT)

ŠIS EKSPLOATAVIMO VADOVAS YRA PRIEINAMAS TIK ANGLŪ KALBA.

- JEI KLIENTO PASLAUGŲ TIEKĒJAS REIKALAUJA VADOVO KITA KALBA – NE ANGLŪ, NUMATYTI VERTIMO PASLAUGAS YRA KLIENTO ATSAKOMYBĖ.
- NEMĖGINKITE ATLIKTI ĮRANGOS TECHNINĖS PRIEŽIŪROS, NEBENT ATSIŽVELGĖTE Į ŠĮ EKSPLOATAVIMO VADOVĄ IR JĮ SUPRATOTE.
- JEI NEATKREIPSITE DĖMESIO Į ŠĮ PERSPĖJIMĄ, GALIMI SUŽALOJIMAI DĖL ELEKTROS ŠOKO,
- MECHANINIŲ AR KITŲ PAVOJŲ PASLAUGŲ TIEKĖJUI, OPERATORIUI AR PACIENTUI.

ADVARSEL
(NO)

DENNE SERVICEHÅNDBOKEN FINNES BARE PÅ ENGELSK.

- HVIS KUNDENS SERVICELEVERANDØR TRENGER ET ANNET SPRÅK, ER DET KUNDENS ANSVAR Å SØRGE FOR OVERSETTELSE.
- IKKE FORSØK Å REPARERE UTSTYRET UTEN AT DENNE SERVICEHÅNDBOKEN ER LEST OG FORSTÅTT.
- MANGLENDE HENSYN TIL DENNE ADVARSELEN KAN FØRE TIL AT SERVICELEVERANDØREN, OPERATØREN ELLER PASIENTEN SKADES PÅ GRUNN AV ELEKTRISK STØT, MEKANISKE ELLER ANDRE FARER.

OSTRZEŻENIE
(PL)

NINIEJSZY PODRĘCZNIK SERWISOWY DOSTĘPNY JEST JEDYNIEM W JĘZYKU ANGIELSKIM.

- JEŚLI DOSTAWCA USŁUG KLIENTA WYMAGA JĘZYKA INNEGO NIŻ ANGIELSKI, ZAPEWNIENIE USŁUGI TŁUMACZENIA JEST OBOWIĄZKIEM KLIENTA.
- NIE PRÓBOWAĆ SERWISOWAĆ WYPOSAŻENIA BEZ ZAPOZNANIA SIĘ I ZROZUMIENIA NINIEJSZEGO PODRĘCZNIKA SERWISOWEGO.
- NIEZASTOSOWANIE SIĘ DO TEGO OSTRZEŻENIA MOŻE SPOWODOWAĆ URAZY DOSTAWCY USŁUG, OPERATORA LUB PACJENTA W WYNIKU PORAŻENIA ELEKTRYCZNEGO, ZAGROŻENIA MECHANICZNEGO BĄDŹ INNEGO.

AVISO
(PT-BR)

ESTE MANUAL DE ASSISTÊNCIA TÉCNICA SÓ SE ENCONTRA DISPONÍVEL EM INGLÊS.

- SE QUALQUER OUTRO SERVIÇO DE ASSISTÊNCIA TÉCNICA, QUE NÃO A GEMS, SOLICITAR ESTES MANUAIS NOUTRO IDIOMA, É DA RESPONSABILIDADE DO CLIENTE FORNECER OS SERVIÇOS DE TRADUÇÃO.
- NÃO TENHA TENTADO REPARAR O EQUIPAMENTO SEM TER CONSULTADO E COMPREENDIDO ESTE MANUAL DE ASSISTÊNCIA TÉCNICA.

O NÃO CUMPRIMENTO DESTES AVISOS PODE POR EM PERIGO A SEGURANÇA DO TÉCNICO, OPERADOR OU PACIENTE DEVIDO A CHOQUES ELÉTRICOS, MECÂNICOS OU OUTROS.

ATENÇÃO
(PT-PT)

ESTE MANUAL DE ASSISTÊNCIA TÉCNICA SÓ SE ENCONTRA DISPONÍVEL EM INGLÊS.

- SE QUALQUER OUTRO SERVIÇO DE ASSISTÊNCIA TÉCNICA, QUE NÃO A GEMS, SOLICITAR ESTES MANUAIS NOUTRO IDIOMA, É DA RESPONSABILIDADE DO CLIENTE FORNECER OS SERVIÇOS DE TRADUÇÃO.
- NÃO TENHA TENTADO REPARAR O EQUIPAMENTO SEM TER CONSULTADO E COMPREENDIDO ESTE MANUAL DE ASSISTÊNCIA TÉCNICA
- O NÃO CUMPRIMENTO DESTES AVISOS PODE COLOCAR EM PERIGO A SEGURANÇA DO TÉCNICO, DO OPERADOR OU DO PACIENTE DEVIDO A CHOQUES ELÉTRICOS, MECÂNICOS OU OUTROS.

ATENȚIE
(RO)

ACEST MANUAL DE SERVICE ESTE DISPONIBIL NUMAI ÎN LIMBA ENGLEZĂ.

- DACĂ UN FURNIZOR DE SERVICII PENTRU CLIENȚI NECESITĂ O ALTĂ LIMBĂ DECÂT CEA ENGLEZĂ, ESTE DE DATORIA CLIENTULUI SĂ FURNIZEZE O TRADUCERE.
- NU ÎNCERCAȚI SĂ REPARAȚI ECHIPAMENTUL DECÂT ULTERIOR CONSULTĂRII ȘI ÎNȚELEGERII ACESTUI MANUAL DE SERVICE.
- IGNORAREA ACESTUI AVERTISMENT AR PUTEA DUCE LA RĂNIREA DEPARATORULUI, OPERATORULUI SAU PACIENTULUI ÎN URMA PERICOLELOR DE ELECTROCUTARE, MECANICE SAU DE ALTĂ NATURĂ.

ОСТОРОЖНО!
(RU)

ДАННОЕ РУКОВОДСТВО ПО ОБСЛУЖИВАНИЮ ПРЕДЛАГАЕТСЯ ТОЛЬКО НА АНГЛИЙСКОМ ЯЗЫКЕ.

- ЕСЛИ СЕРВИСНОМУ ПЕРСОНАЛУ КЛИЕНТА НЕОБХОДИМО РУКОВОДСТВО НЕ НА АНГЛИЙСКОМ, А НА КАКОМ-ТО ДРУГОМ ЯЗЫКЕ, КЛИЕНТУ СЛЕДУЕТ САМОСТОЯТЕЛЬНО ОБЕСПЕЧИТЬ ПЕРЕВОД.
- ПЕРЕД ОБСЛУЖИВАНИЕМ ОБОРУДОВАНИЯ ОБЯЗАТЕЛЬНО ОБРАТИТЕСЬ К ДАННОМУ РУКОВОДСТВУ И ПОЙМИТЕ ИЗЛОЖЕННЫЕ В НЕМ СВЕДЕНИЯ.
- НЕСОБЛЮДЕНИЕ ТРЕБОВАНИЙ ДАННОГО ПРЕДУПРЕЖДЕНИЯ МОЖЕТ ПРИВЕСТИ К ТОМУ, ЧТО СПЕЦИАЛИСТ ПО ОБСЛУЖИВАНИЮ, ОПЕРАТОР ИЛИ ПАЦИЕНТ ПОЛУЧАТ УДАР ЭЛЕКТРИЧЕСКИМ ТОКОМ, МЕХАНИЧЕСКУЮ ТРАВМУ ИЛИ ДРУГОЕ ПОВРЕЖДЕНИЕ.

UPOZORENJE
(SR)

OVO SERVISNO UPUTSTVO JE DOSTUPNO SAMO NA ENGLJESKOM JEZIKU.

- AKO KLIJENTOV SERVISER ZAHTEVA NEKI DRUGI JEZIK, KLIJENT JE DUŽAN DA OBEZBEDI PREVODILAČKE USLUGE.
- NE POKUŠAVAJTE DA OPRAVITE UREĐAJ AKO NISTE PROČITALI I RAZUMELI OVO SERVISNO UPUTSTVO.
- ZANEMARIVANJE OVOG UPOZORENJA MOŽE DOVESTI DO POVREĐIVANJA SERVISERA, RUKOVAOCA ILI PACIJENTA USLED STRUJNOG UDARA ILI MEHANIČKIH I DRUGIH OPASNOSTI.

UPOZORNENIE
(SK)

TENTO NÁVOD NA OBSLUHU JE K DISPOZÍCII LEN V ANGLIČTINE.

- AK ZÁKAZNÍKOV POSKYTOVATEĽ SLUŽIEB VYŽADUJE INÝ JAZYK AKO ANGLIČTINU, POSKYTNUTIE PREKLADATEĽSKÝCH SLUŽIEB JE ZODPOVEDNOSŤOU ZÁKAZNÍKA.
- NEPOKÚŠAJTE SA O OBSLUHU ZARIADENIA SKÔR, AKO SI NEPREČÍTATE NÁVOD NA OBLUHU A NEPOROZUMIETE MU.
- ZANEDBANIE TOHTO UPOZORNENIA MÔŽE VYÚSTIŤ DO ZRANENIA POSKYTOVATEĽA SLUŽIEB, OBSLUHUJÚCEJ OSOBY ALEBO PACIENTA ELEKTRICKÝM PRÚDOM, DO MECHANICKÉHO ALEBO INÉHO NEBEZPEČENSTVA.

- ATENCION**
(ES)
- ESTE MANUAL DE SERVICIO SOLO EXISTE EN INGLES.
- SI ALGUN PROVEEDOR DE SERVICIOS AJENO A GEMS SOLICITA UN IDIOMA QUE NO SEA EL INGLES, ES RESPONSABILIDAD DEL CLIENTE OFRECER UN SERVICIO DE TRADUCCION
 - NO SE DEBERA DAR SERVICIO TECNICO AL EQUIPO, SIN HABER CONSULTADO Y COMPRENDIDO ESTE MANUAL DE SERVICIO
 - LA NO OBSERVANCIA DEL PRESENTE AVISO PUEDE DAR LUGAR A QUE EL PROVEEDOR DE SERVICIOS, EL OPERADOR O EL PACIENTE SUFRAN LESIONES PROVOCADAS POR CAUSAS ELÉCTRICAS, MECÁNICAS O DE OTRA NATURALEZA.
- VARNING**
(SV)
- DEN HÄR SERVICEHANDBOKEN FINNS BARA TILLGÄNGLIG PÅ ENGELSKA.
- OM EN KUNDS SERVICETEKNIKER HAR BEHOV AV ETT ANNAT SPRÅK ÄN ENGELSKA ANSVARAR KUNDEN FÖR ATT TILLHANDAHÅLLA ÖVERSÄTTNINGSTJÄNSTER.
 - FÖRSÖK INTE UTFÖRA SERVICE PÅ UTRUSTNINGEN OM DU INTE HAR LÄST OCH FÖRSTÅR DEN HÄR SERVICEHANDBOKEN.
 - OM DU INTE TAR HÄNSYN TILL DEN HÄR VARNINGEN KAN DET RESULTERA I SKADOR PÅ SERVICETEKNIKERN, OPERATÖREN ELLER PATIENTEN TILL FÖLJD AV ELEKTRISKA STÖTAR, MEKANISKA FAROR ELLER ANDRA FAROR.
- OPOZORILO**
(SL)
- TA SERVISNI PRIROČNIK JE NA VOLJO SAMO V ANGLEŠKEM JEZIKU.
- ČE PONUDNIK STORITVE STRANKE POTREBUJE PRIROČNIK V DRUGEM JEZIKU, MORA STRANKA ZAGOTOVITI PREVOD.
 - NE POSKUŠAJTE SERVISIRATI OPREME, ČE TEGA PRIROČNIKA NISTE V CELOTI PREBRALI IN RAZUMELI.
 - ČE TEGA OPOZORILA NE UPOŠTEVATE, SE LAHKO ZARADI ELEKTRIČNEGA UDARA, MEHANSKIH ALI DRUGIH NEVARNOSTI POŠKODUJE PONUDNIK STORITEV, OPERATER ALI BOLNIK.
- DİKKAT**
(TR)
- BU SERVİS KILAVUZUNUN SADECE İNGİLİZCESİ MEVCUTTUR.
- EĞER MÜŞTERİ TEKNİSYENİ BU KILAVUZU İNGİLİZCE DIŞINDA BİR BAŞKA LİSANDAN TALEP EDERSE, BUNU TERCÜME ETTİRMEK MÜŞTERİYE DÜŞER.
 - SERVİS KILAVUZUNU OKUYUP ANLAMADAN EKİPMANLARA MÜDAHALE ETMEYİNİZ.
 - BU UYARIYA UYULMAMASI, ELEKTRİK, MEKANİK VEYA DİĞER TEHLİKELERDEN DOLAYI TEKNİSYEN, OPERATÖR VEYA HASTANIN YARALANMASINA YOL AÇABİLİR.

1 - EQUIPMENT SPECIFICATIONS

IMPORTANT

This manual now contains the procedure for installing magnet monitor onto older magnets for Excite HD upgrades. Refer to Appendix B for instructions on how to connect the pressure transducer to S-Series Magnets (S2 through CX).

1-1 Equipment Specifications for Granite Magnet Monitor


Dimensions: 9.4 in. (23.9 cm) H x 18.6 in. (47.3 cm) W x 7.0 in. (17.8 cm) D

Weight: 26 pounds (11.8 kg)

Input Power: 100/120 or 200/220 VAC, 50 or 60 Hz @ 0.5 A

Gauss Limit: 100 Gauss or less

Center of Gravity: See Illustration 1-1.

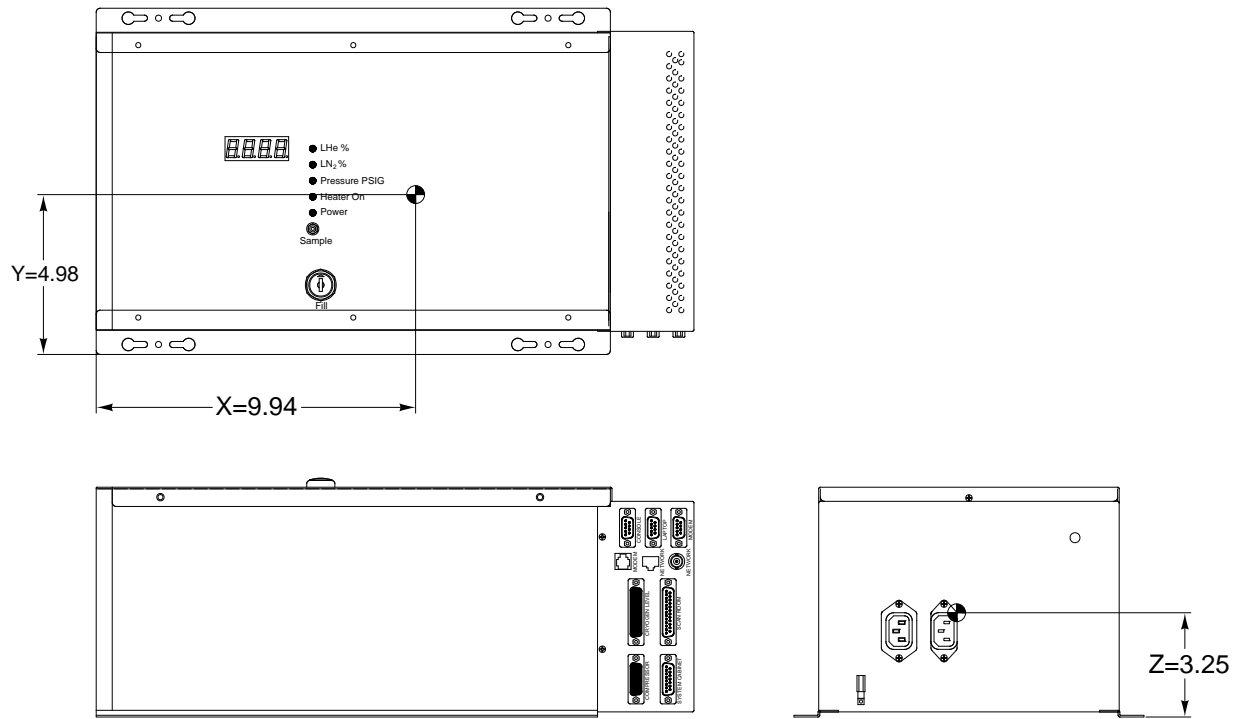

CENTER OF GRAVITY
ILLUSTRATION 1-1

Temperature:

Operating: 0 to 40 degrees C

Non-Operating: -34 to 50 degrees C

Temperature Change: 5 degrees C per hour

Humidity:


Operating: 10% to 80% noncondensing

Non-Operating: 10% to 90% noncondensing

Altitude: -30 to 2,133 meters

Mounting: Wall mounted

Service Clearance: See Illustration 1-2.

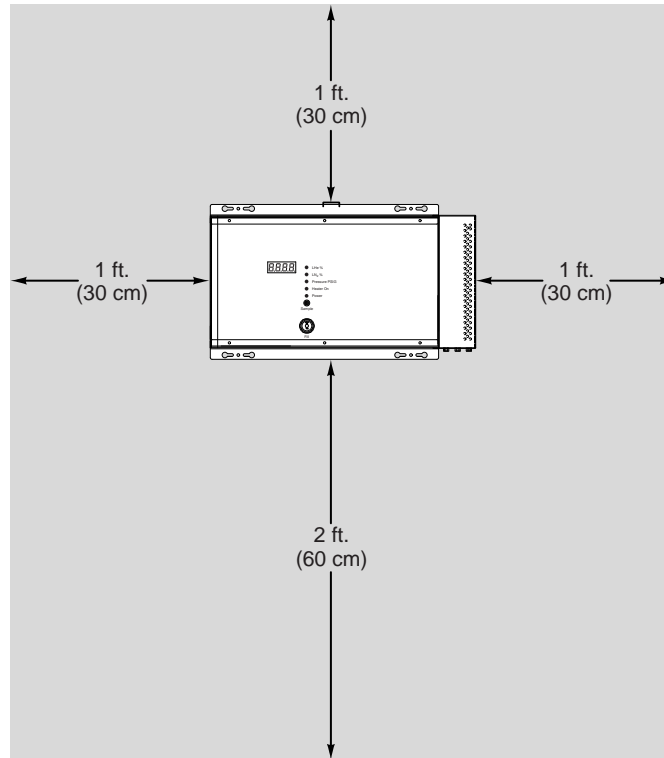

SERVICE CLEARANCE
ILLUSTRATION 1-2

1-2 Equipment Specifications for Advantech Magnet Monitor


Dimensions: 10.0 in. (25.4 cm) H x 15.5 in. (39.4 cm) W x 7.0 in. (17.8 cm) D

Weight: 26 pounds (11.8 kg)

Input Power: 100/120 or 200/220 VAC, 50 or 60 Hz @ 0.5 A

Gauss Limit: 50 Gauss or less

Center of Gravity: See Illustration 1-3.

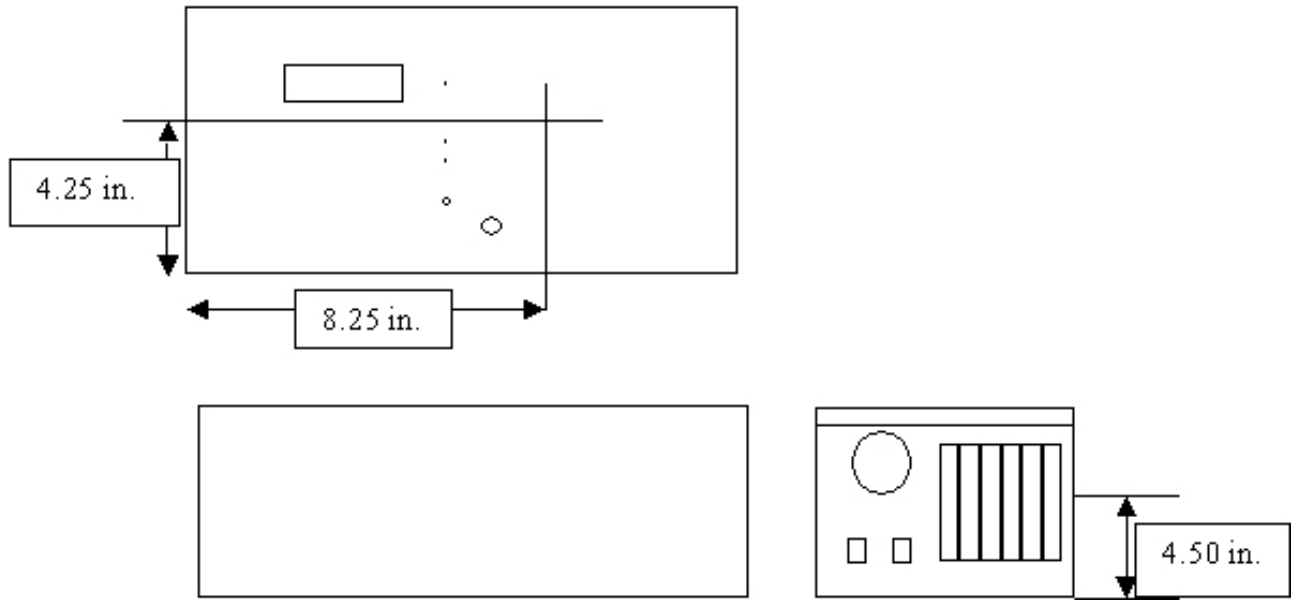

CENTER OF GRAVITY
ILLUSTRATION 1-3

Temperature:

Operating: 0 to 40 degrees C
Non-Operating: -34 to 50 degrees C

Temperature Change: 5 degrees C per hour


Humidity:

Operating: 10% to 80% noncondensing
Non-Operating: 10% to 90% noncondensing

Altitude: -30 to 2,133 meters

Mounting: Wall mounted

Service Clearance: See Illustration 1-4.

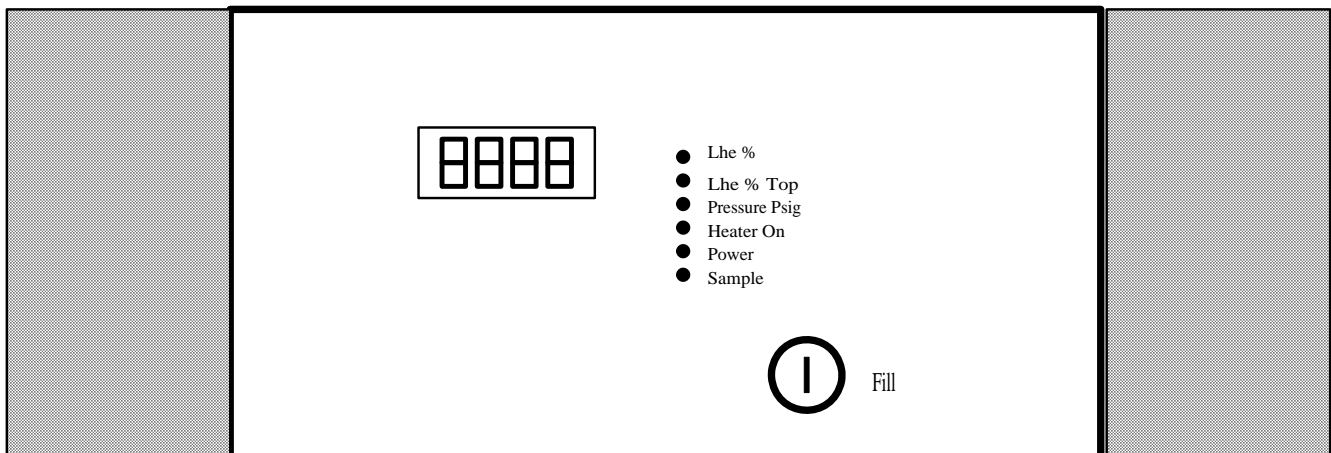

SERVICE CLEARANCE
ILLUSTRATION 1-4

TABLE 1-1
COMPRESSOR CONNECTION

J9 Pin Number	Definition	Voltage (DC)
1	Water Flow	0 to 5V
2	Water Temperature	0 to 5V
3	Water Flow DC Supply	+12V
4	Water Flow Ground	Ground
5	Compressor Fuse Off	+24V
6	Helium Temperature High	+24V
7	Helium Pressure Low	+24V
8	Compressor On	+24V
9	Klixon Error	+24V
10	Compressor Reset	+24V
11	+24 VDC Power	+24V
12	+24 VDC Ground	Ground
13	Not Connected	
14	Water Flow #2	0 to 5V
15	Water Temperature #2	0 to 5V

TABLE 1-2
SYSTEM CABINET CONNECTOR

J10 Pin Number	Definition	Voltage (DC)	Maximum Current
1	Not Connected		
2	Not Connected		
3	Not Connected		
4	Not Connected		
5	Not Connected		
6	RF Unblank	+5V	250 mA
7	RF Unblank Return	+5V	250 mA
8	Key		
9	Ground	Ground	250 mA
10	System Power	+5V	250 mA
11	System Power Return	+5V	250 mA
12	Not Connected		
13	Not Connected		
14	Not Connected		
15	Not Connected		

TABLE 1-3
SCAN ROOM CONNECTOR

J8 Pin Number	Definition	Voltage (DC)	Maximum Current
1	First Stage Shield II Diode Source +		10 μ A
2	First Stage Shield II Diode Source -		10 μ A
3	First Stage Shield II Diode Signal +	0 to 2.5V	10 μ A
4	First Stage Shield II Diode Signal -	0 to 2.5V	10 μ A
5	-12 VDC Power Supply	-12V	250 mA
6	-12 VDC Ground	Ground	250 mA

J8 Pin Number	Definition	Voltage (DC)	Maximum Current
7	Vessel Pressure	1 to 5V	50 mA
8	Differential Pressure	1 to 5V	50 mA
9	+12 VDC Power Supply	+12V	250 mA
10	+12 VDC Ground	Ground	250 mA
11	First Stage Diode Source +		10 μ A
12	First Stage Diode Source -		10 μ A
13	First Stage Diode Signal +	0 to 2.5V	10 μ A
14	First Stage Diode Signal -	0 to 2.5V	10 μ A
15	Second Stage Diode Source +		10 μ A
16	Second Stage Diode Source -		10 μ A
17	Second Stage Diode Signal +	0 to 2.5V	10 μ A
18	Second Stage Diode Signal -	0 to 2.5V	10 μ A
19	Reed Switch - Magnet On Sense	TTL Logic	5 mA
20	Reed Switch Common	Ground	5 mA
21	Atmospheric Pressure	1 to 5V	250 mA
22	Second Stage Shield II Diode Source +		10 μ A
23	Second Stage Shield II Diode Source -		10 μ A
24	Second Stage Shield II Diode Signal +	0 to 2.5V	10 μ A
25	Second Stage Shield II Diode Signal -	0 to 2.5V	10 μ A

2 - INTRODUCTION

2-1 Overview

The Magnet Monitor consists of hardware and software used to:

- Measure helium vessel level
- Measure helium vessel pressure
- Provide helium vessel pressure control on LCC magnets

2-2 System Requirements

The Magnet Monitor works with Signa 3.x, 4.x, 5.x, LX and Twin systems with the following magnets: S-II, S-III, S-III Magnishield, S-IV, S-V, S-X, SXc, Cx, LCC, LCC 3T, and Max

2-3 Document Application

The current revision of this document applies to the following combination(s):

2-3-1 Core Magnet with Sumitomo Compressor

This system consists of an LCC magnet with a Sumitomo compressor. There is an external water flow/temperature sensor connected to the water source for the Sumitomo compressor. The water flow/temperature information is sent to the Magnet Monitor.

2-3-2 S-Series Magnets through CX

System consists of older style magnets that are upgraded to Excite HD.

3 - TOOLS AND EQUIPMENT

3-1 Required Tools/Equipment

When installing the Magnet Monitor, the following tools are required:

- Duct tape (if performing installation without a partner)
- Nonmagnetic Adjustable Wrench
- Nonmagnetic 9/16-in. Combination Wrench
- Nonmagnetic 3/16-in. Flat-blade Screwdriver
- Ruler
- Utility Knife
- Nut Drivers, 3/16 in. and 1/4 in. sizes
- Helium Resistance Box (46-265286G1)
- Digital Volt Meter (DVM)
- No. 2 Phillips Screwdriver
- Static Wrist Strap
- Shallow Container (to catch water after disconnecting the hose to the Shield Cooler Compressor)
- Any tools and hardware needed for mounting the Magnet Monitor to the wall
- Laptop Computer (minimum requirement is Windows95 with a modem and Internet Explorer or Netscape Navigator):
 - with modem and cables (46-301708G9)
 - without modem (46-301708G10)
- Serial Interface Cable, 9-pin Female to 9-pin Female (for configuring the Magnet Monitor with the laptop computer)

3-2 Optional Parts

The Magnet Monitor provides most of the parts required during the installation. However, one or more of the following parts may need to be obtained, depending on the system requirements:

- Additional cables (for sites where the Penetration Panel is more than 60 ft. away from the Magnet Monitor)
 - 25-Pin Male-Female, 10 ft. Cable (46-271601G19)
 - 25-Pin Male-Female, 20 ft. Cable (46-271601G21)
 - 25-Pin Male-Female, 30 ft. Cable (46-271601G23)
 - 25-Pin Male-Female, 40 ft. Cable (46-271601G25)
 - 25-Pin Male-Female, 50 ft. Cable (46-271601G27)
- Multiplexer with four station lines
 - 115 VAC Power Supply (46-328475P1)
 - 220 VAC, 50 Hz Power Supply (46-328475P3)
- Multiplexer with eight station lines
 - 115 VAC Power Supply (46-328475P2)
 - 220 VAC, 50 Hz Power Supply (46-328475P4)

3-3 Magnet Monitor Description

The Magnet Monitor is described in the following table.

TABLE 3-1
MAGNET MONITOR STRUCTURED KITS

Item	Description	Part Number	
		Fixed and Transportable/ Relocatable Unit	Mobile Unit
1	Magnet Monitor- Granite	2209000	2209000
2	Magnet Monitor- Advantech	2219400	2219400
2	LCC Install Cable Kit	2221624	2221624-2
3	Twin Install Cable Kit	2304057	2304057-2
4	Magnet Monitor Operator's Manual	2229090-100	2229090-100
5	Magnet Monitor Hardware Installation Manual	2230681	2230681
6	Uninterruptable Power Supply (UPS) Option for Advantech only	E4504AG	Included
7	Remote Alarm Box Option for Advantech only	E4054AH	Included

4 - INSTALLATION PROCEDURES

Note

Most of the hardware installation can be performed while the customer is using the system. The Magnet Room is needed for only about 30 minutes to install cables and sensors. The three to four hours of installation time in the Computer Room do not require the system to be down. The Magnet Monitor can be shut down and worked on while the Signa system is running.

4-1 Magnet Room Attachments

The Pressure Transducer, Shield Diodes, and Reed Switch generate signals that are incorporated into a cable that enters the Magnet Room through the Penetration Panel. These signals aid in assessing boil off, magnet quench, and proper maintenance of vessel pressures before, during, and after a helium fill.

4-1-1 Before Beginning


The following tools are required:

- Duct tape (if performing installation without a partner)
- Nonmagnetic adjustable wrench
- Nonmagnetic, 3/16-in. flat-blade screwdriver
- Nonmagnetic, 3/16-in. nut driver
- 25-Pin “D” Punch (if a spare 25-pin connector opening is not available at site Penetration Panel)

The cable runs installed in the following procedures are diagrammed in Illustrations 8-1 through 8-4. The instructions are easier to follow if this System Interconnect Diagram and the associated table are referenced.

4-1-2 Penetration Panel Connection

1. For Runs 825 and 829: Locate a space in the Penetration Panel that will fit a 25-Pin "D" Connector (J48 should be available at most sites).
 - a. If a space is available, place a piece of duct tape across the blank plate to keep it from falling between the walls. Using a flat-blade screwdriver, remove the screws holding the blank plate.
 - b. If no space is available, use a 25-Pin "D" Punch to make an opening in the Penetration Panel.
2. Inside the Magnet Room:
 - a. Put the Filter Gasket (46-221754P3) onto the male side of the Penetration Panel Filter (46-271920P3).
 - b. Place the filter into the Penetration Panel with the female side toward the Magnet Room. (See Illustration 4-1.)

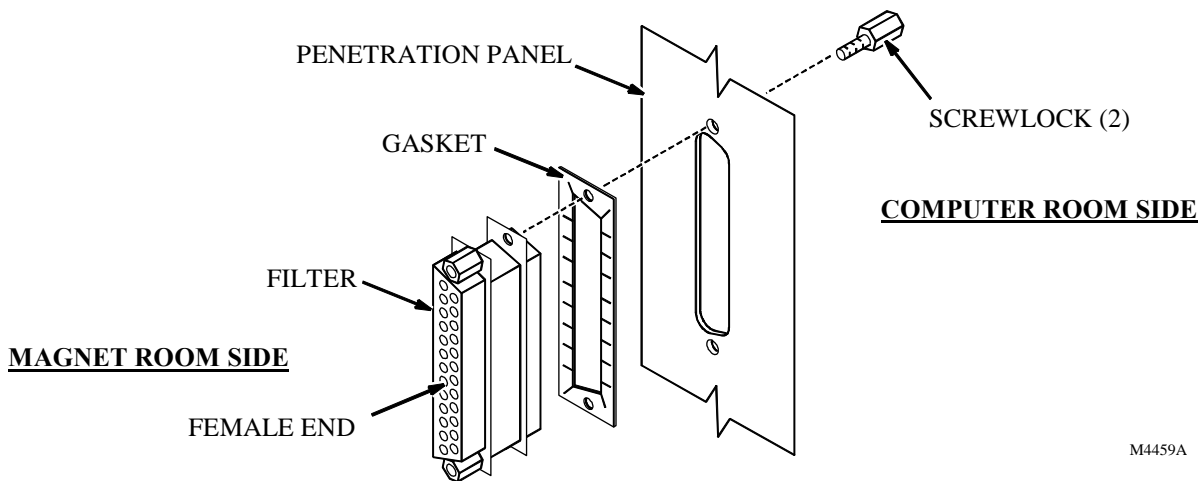

PENETRATION PANEL CONNECTIONS
ILLUSTRATION 4-1

M4459A

- c. Place a piece of duct tape across the filter to hold it in place.
- d. From the other side of the Penetration Panel, use a pair of screwlocks without washers to secure the filter to the panel.

4-1-3 Connecting Magnet Room Cables (All Systems)

Notes

If the cable in the following step is routed before it is connected, be sure to pay attention to gender of the connectors.

Remove the twist locks around the cables before connecting cables in the Magnet Room.

1. Run 829, 60 ft. Cable:
 - a. From the Magnet Room, use a nonmagnetic flat-blade screwdriver to connect the PP1-J48 end to the Penetration Panel Filter.
 - b. Route this cable from the Penetration Panel to the rear pedestal.


Note

Do not coil the cables under the rear pedestal. Serpentine any excess cable in the cable trough so it will not interfere with scanning.

- c. Run 830 (Illustration 4-2):

IMPORTANT

When installing the magnet monitor onto an older-style magnet, do NOT use Run 830 provided in the original Magnet Monitor Cable Kit. Use Cable 2108726 provided in the Magnet Monitor Upgrade Kit for non-LCC systems, P/N 5127838. Connections are the same as noted in steps a through d that follow, except the 9-pin Female Cable connects to the interface cable that routes to the Coldhead Sleeve. This cable was provided via FMI 60374.

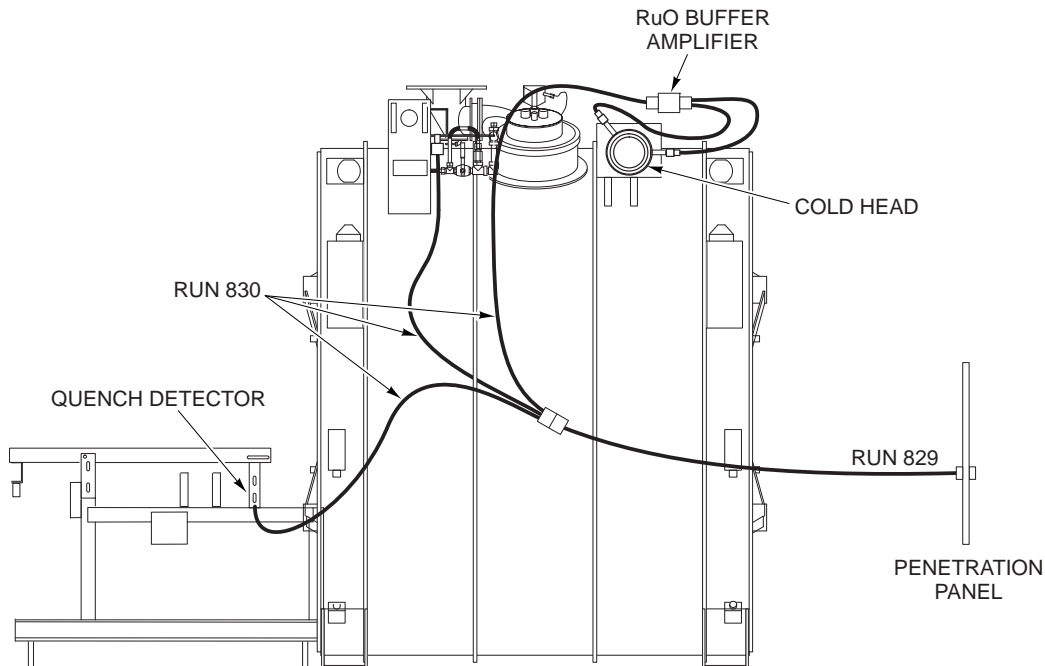


LCC MAGNET ROOM CABLING
ILLUSTRATION 4-2

- a. Route the MS1-A5-A2-J1 Connector along the magnet enclosure to the top of the magnet and connect to the Pressure Transducer. Wrap the cable using cable ties when necessary.
- b. Install the Reed Switch lead per Section 4-1-4.
- c. Connect the MS1-A2-J2 connector on the RuO Buffer Amplifier.
- d. Connect the MS1-FJ2 connector of cable Run 830 to Run 829. Use two Cable Connector Joiners (46-265067P1) to secure the cable connectors to each other.

Note

Do not coil the cables under the rear pedestal. Serpentine any excess cable in the cable trough so it will not interfere with scanning.

4. Run 832: Connect to the Coldhead Sleeve and RuO Buffer Amplifier. (See Illustration 4-2.)
5. Run 828 (Illustration 4-3):
 - a. Connect the PP1-J10 connector to J10 on the Penetration Panel.
 - b. Connect the MS1-A3-A1-P403 connector to P403 on the magnet.

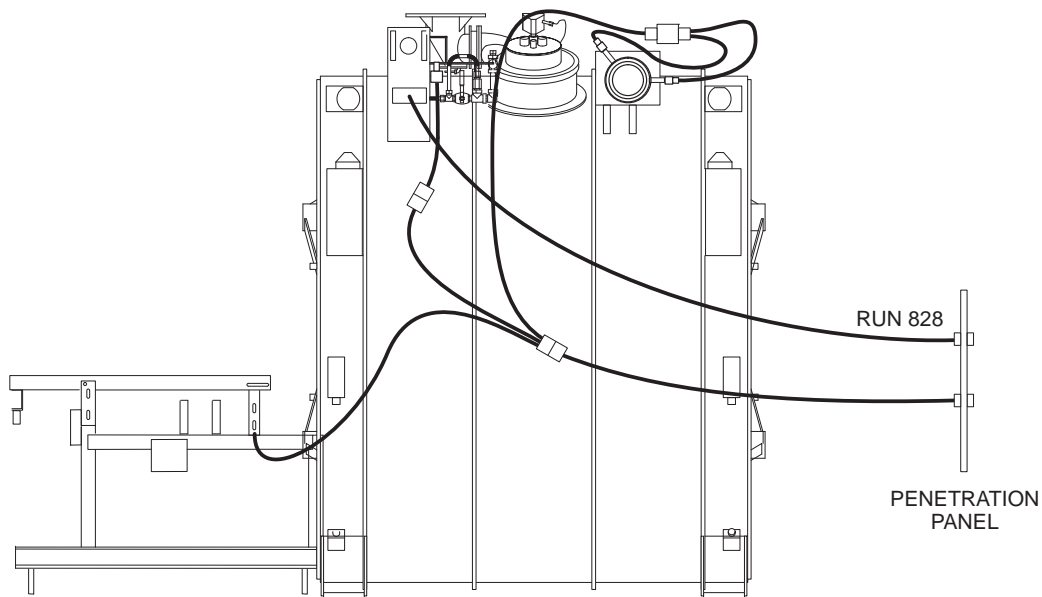


SENSOR CABLING
ILLUSTRATION 4-3

4-1-4 Connecting Reed Switch

Note

The Reed Switch (2108790) is pre-assembled to Run 830, the Magnet Room Interface Cable (2222797). Discovery and Optima systems use the Magnet Room Interface Cable (5352964), which does **not** have a Reed Switch. In an upgrade scenario, if the site already has a reed switch it can be installed in place of the 5352964.

1. Position the Reed Switch where it detects the magnetic field. The contacts are open when the Reed Switch detects a magnetic field. (Usually the switch can be placed vertically on one of the support struts on the rear pedestal as shown in Illustration 4-4.)
2. Wrap two Cable Ties (46-208758P3) around the top and bottom of the Reed Switch and the support strut to hold the switch in place. (See Illustration 4-4.)
3. Use an additional cable tie to secure the cable to the strut to prevent the wires from being pulled out of the switch. Move the ends of the cable ties to the back of the strut.

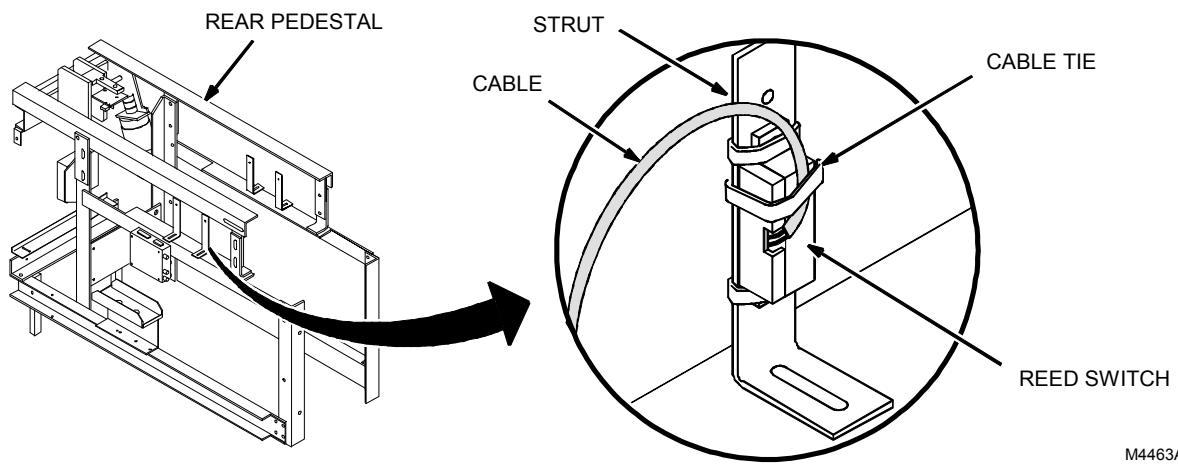


M4463A

REED SWITCH PLACEMENT
ILLUSTRATION 4-4

4-1-5 Attaching Preamp to Magnet

The RUO Buffer Preamp is part of the magnet collector. It must be positioned away from potential moisture to prevent failures.

1. Install Velcro® strips onto the magnet in the location as shown in Illustration 4-5.

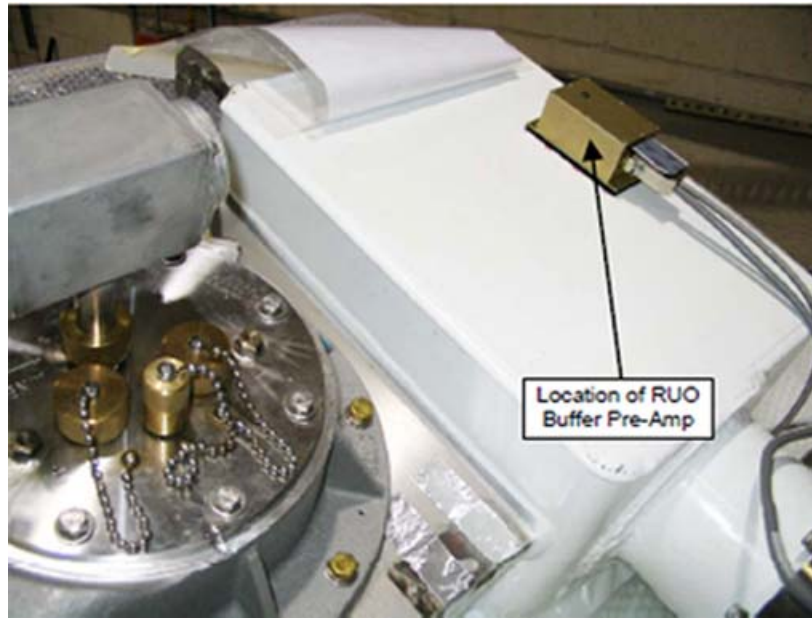


LOCATION OF RUO BUFFER PREAMP ON COLDHEAD BOX
ILLUSTRATION 4-5

2. Place Preamp onto Velcro.
3. Connect the cable from the coldhead and sleeve to the Preamp box input.
4. Connect the cable from Run 830 to the Preamp box output. (See Illustration 4-2.)


4-2 Installing Magnet Monitor (All Systems)

4-2-1 Physical Installation

1. Anchor the Magnet Monitor to the wall using all applicable codes for the site.
2. Run 825:
 - a. In the computer room, connect the PP1-J48 end to the Penetration Panel Filter (J48).
 - b. Route the cable to the Magnet Monitor and connect to J8. (See Illustration 4-6A for the connections to the **Granite** Magnet Monitor. If connecting to the **Advantech** Magnet monitor, see Illustration 4-6B.)

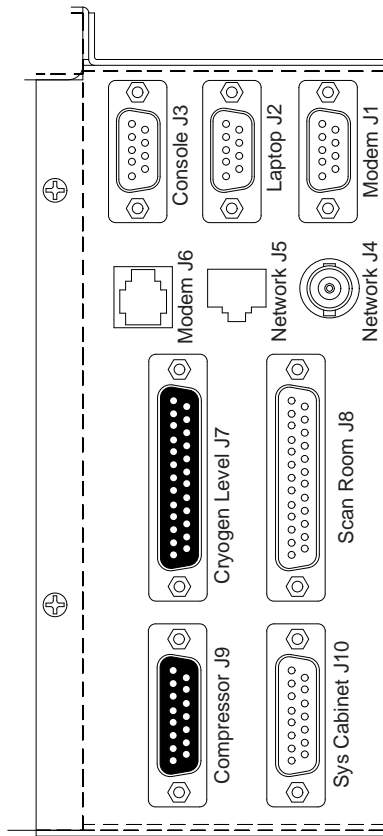

MAGNET MONITOR INTERFACE PANEL
ILLUSTRATION 4-6A

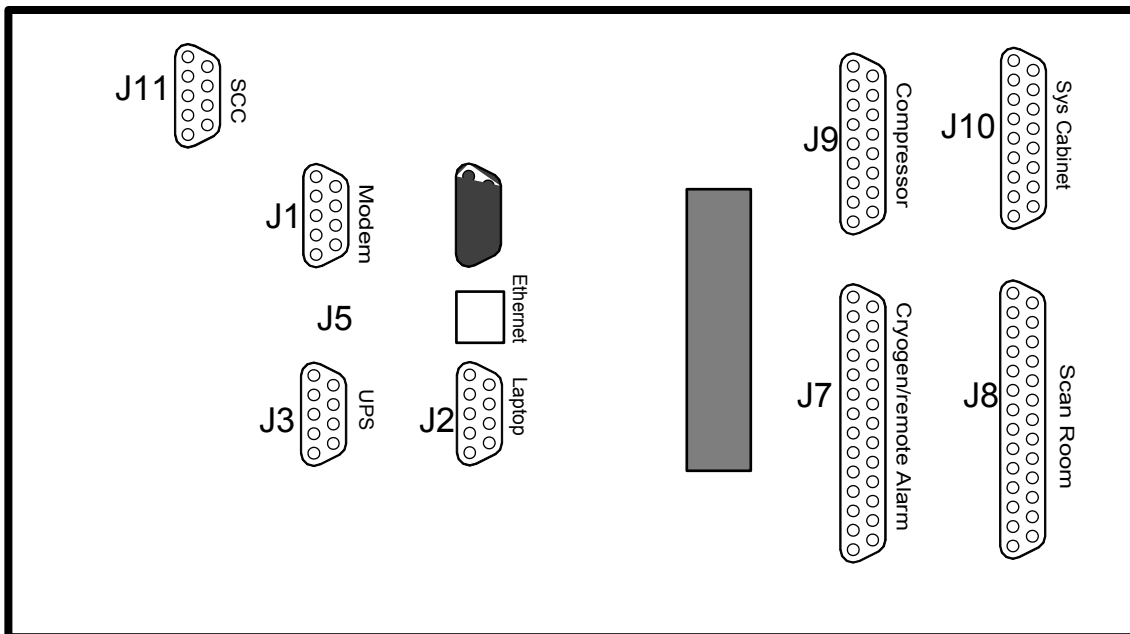

MAGNET MONITOR INTERFACE PANEL
ILLUSTRATION 4-6B

3. Runs 831 and 824:
 - a. Connect the MSM1-A1-J7 end of Run 831 to the Magnet Monitor.
 - b. Connect the FJ1 end of Run 831 to Run 824. Use two cable connector joiners (46-265067P1) to secure the cable connectors to each other. Wrap the cables using cable ties when necessary.
 - c. Connect the PP1-J10 end of Run 824 to the Penetration Panel.
 - d. Connect cable Run 825 between MSM1-A1-J8 to the Penetration Panel J48.

IMPORTANT

To reset or power down the Magnet Monitor, follow the steps below to properly shut down the software.

- Press and hold the [Sample] button until **halt** appears on the display.
 - Release the [Sample] button, immediately press and release it again. **oooo** (four lower case o's) should appear on the display.
 - Wait for at least **90 seconds**, then reset or power down the Magnet Monitor.
4. Plug the Magnet Monitor AC power cord into the receptacle using the appropriate power cord. Ensure that the power remains available at this receptacle 24 hours/day to assure constant monitoring, and pressure control of the magnet.

Note

If preferred, the Magnet Monitor can be plugged into a 220 VAC outlet. For such operation, obtain a power cord with the appropriate connector, and perform the following procedure to change the operating voltage of the Magnet Monitor.

4-2-2 Changing Operating Voltage of Magnet Monitor



Do not turn on the Magnet Monitor until the proper input voltage is set.

4-2-2-1 Changing Operating Voltage for Granite Magnet Monitor

Note

This procedure is only required to change the operating voltage of the Magnet Monitor from 110 VAC to 220 VAC.

1. Unplug the Magnet Monitor.
2. Remove the four Phillips screws that hold the rear connector housing to the main chassis.
3. Separate the rear connector housing from the chassis and leave all cables attached.
4. The voltage selector switch for the computer power supply is located on the rear, off the main chassis. There is a bright yellow caution sticker next to the switch that reads, "CAUTION Additional line voltage selector switch in unit." Slide this switch to the desired voltage level setting.

- Another power supply is located on the inside of the rear connector housing. There is another bright yellow caution sticker that also reads, "CAUTION Additional line voltage selector switch in unit." Slide this switch to the desired voltage level setting.

Note

Both of the line voltage selector switches must display the same voltage setting and must match the rating of the input supply voltage.


- Reinstall the rear connector housing using the four Phillips screws.
- Using an appropriate line cord, plug the Magnet Monitor to a power source that matches the voltage rating selected.

4-2-2-2 Changing Operating Voltage for an Advantech Magnet Monitor

There is a 115 VAC/220 VAC switch located above the input power cable. Select 220 VAC if needed for the site.

4-2-3 Configuring Magnet Monitor for External Modem (Granite Unit Only)

- Remove the modem board, located in Slot 3, from the magnet monitor.
- Configure the jumpers on the General I/O Board as shown in Illustration 4-7.

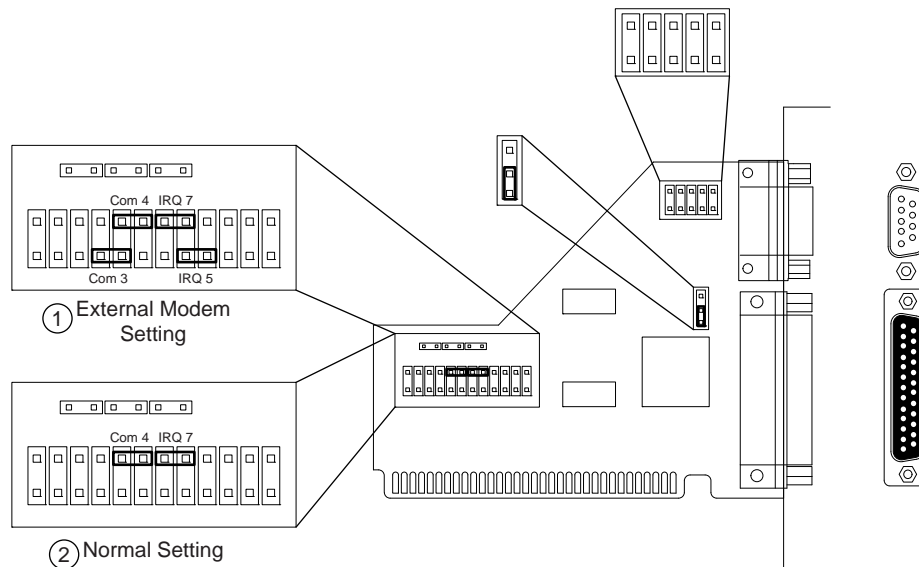

GENERAL I/O BOARD JUMPER SETTINGS
ILLUSTRATION 4-7

4-2-4 Installing External Modem (Advantech Unit Only)

- Place the Global Modem on the top of the Magnet Monitor. Use existing Velcro to secure the modem to the Magnet Monitor.
- Use the proper power adapters for the power connection from the wall to the power supply. (The modem power supply will work for any voltage and frequency.)
- Connect cable Run 940, provided with the cable kit between Magnet Monitor J1 and Modem serial connector.

4-3 Sumitomo Shield Cooler Sensor Installation (Core and Twin with TGWC Only)

The Water Flow and Temperature Sensor provides proportional, linear output for both water flow and water temperature. The viewing window on the sensor provides a visual measurement for water flow.

4-3-1 Before Beginning

The following tools are required:

- 3/16-in. flat-blade screwdriver
- Shallow container to hold water

The following parts from the Magnet Monitoring Kit:

- One of the 15-Pin Female-Male, 60 ft. Cables (46-271603G29)
- Water Flow and Temperature Sensor (46-320835P1). Screws are provided to surface mount the sensor if desired.
- Compressor Interface Cable (46-328578P1)
- One Restricted Label (46-301402P1)

The cable runs installed in the following procedures are diagrammed in Illustrations 8-1 through 8-4. The instructions are easier to follow if the System Interconnect Diagram and the associated table are referenced.



Use caution when attaching the hoses to the Water Flow and Temperature sensor to prevent injury. Sliding the hose onto the fittings does require force.

4-3-2 Attaching Water Flow and Temperature Sensor

Notes

Screws are supplied with the water flow and temperature sensor for mounting to the floor or wall. If it is practical to mount the sensor, place it within 24 inches (61 cm) of the compressor.


The Magnet Monitoring Kit contains a 24 in. (61 cm) hose that is 1/2 in. (13 mm) in diameter. If the site has deviated from the 1/2 in. hose barb, the plumbing will need to be modified to connect to the Water Flow and Temperature Sensor. Obtain the necessary parts locally.

1. Turn off power to the Shield Cooler Compressor.
2. Shut off the water flow to the Shield Cooler Compressor. Be sure to shut off both the supply and return lines to minimize water spillage.



If water from the shield cooler spills, clean it up following hospital procedures, as the water may contain chemicals, such as glycol.

3. Disconnect the inlet water line to the Shield Cooler Compressor, and place a container under the line to catch any water that may run off.
4. Connect the 24 in. hose to the water inlet on the Shield Cooler Compressor and secure with a hose clamp. (See Illustration 4-8.)

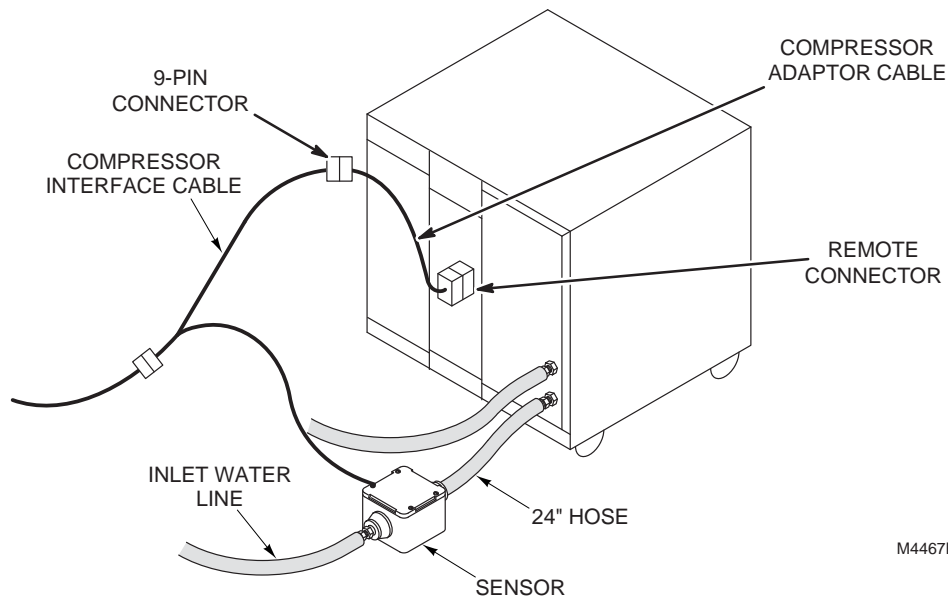

SUMITOMO COMPRESSOR CONNECTIONS
ILLUSTRATION 4-8

5. Connect the other end of the 24 in. hose to the Sensor and secure with a hose clamp. Make sure the Water Direction Arrow is **pointing toward** the compressor inlet.
6. Connect the inlet water line to the inlet of the Water Flow and Temperature Sensor. Secure with a hose clamp.
7. Turn on the water supply and check all fittings for leaks. Repair leaks as necessary. Record water flow reading in the site log using the visual indicator. If practical, use the supplied screws to wall mount the Water Flow and Temperature Sensor.
8. Run 827:
 - a. Connect the MS5-A5-A1-J1 connector to the Water Flow and Temperature Sensor. Align the key on the connector and tighten the screw collar.
 - b. Connect the FJ4 connector on Run 827 to Run 833. Use two Cable Connector Joiners (46-265067P1) to secure the cable connectors to each other. (See Illustration 4-8.)
 - c. Connect the FJ3 connector on Run 827 to Run 826. Use two Cable Connector Joiners (46-265067P1) to secure the cable connectors to each other.
9. Run 833: Connect the MS5-A1-A6-JR end of this cable to the remote connector on the Sumitomo compressor.
10. Run 826: Route the cable to the Magnet Monitor and connect at J9.
11. Turn on power to the Shield Cooler Compressor.

4-4 Installing Magnet Monitor UPS (Optional)

Notes

The wiring diagram of the Magnet Monitor UPS is found in Illustration 8-4.

The Magnet Monitor AC output power is routed through the E-stop circuitry. If the E-stop circuit is not complete at this time, it may need to be by-passed.

1. There is an AC jumper between J1 and J2 in the MDP. Remove this jumper. Locate labels **UPS In** and **UPS Out**. Place the UPS In label over outlet J2. Place the UPS Out label over outlet J1.
2. Position the UPS near the Main Disconnect Panel and the Magnet Monitor. Connect the power cable that is provided with the UPS to the MDP AC outlet labeled *UPS Out*. The cable must be routed through the bottom of the MDP Cabinet. (See Illustration 4-9.)



MDP CONNECTIONS
ILLUSTRATION 4-9

3. Find cable Run 939 provided with the Magnet Monitoring Cable Kit. Connect this cable from the UPS output to the MDP AC outlet labeled *UPS Input*. The AC outlet in the MDP is a Male connector. The cable must be routed through the bottom of the MDP Cabinet.
4. Connect the Magnet Monitor AC power cable to the MDP AC outlet labeled *Magnet Monitor Output*. This cable must be routed through the bottom of the MDP Cabinet.
5. Connect the serial cable connector J3, provided with the UPS to the Magnet Monitor, to the UPS Comm Port connector.

6. Connect the customer phone line to the UPS **phone in** connection. Run the customer-supplied phone line from the UPS **phone out** connection to the global modem.
7. Connect the provided strain reliefs for the power cables that are routed through the MDP panel. They are included in the Magnet Monitor Cable Kit.

4-5 Magnet Monitor Calibration and Final Installation


Note

The sensor I/O board in the **Granite** Magnet Monitor is calibrated at the factory. This procedure should only be done once a year, or if the calibration of the helium reading circuit is questionable. The sensor I/O board in the **Advantech** Magnet Monitor has no adjustments. No calibration is required, but the helium reading checks should be performed every year.

4-5-1 Calibrating Magnet Monitor Helium Level Meter

4-5-1-1 Calibrating Granite Magnet Monitor


1. In the Magnet Room, remove Run 828 from P403 (MS1-A3-A1-P403).
2. On the connector of Run 828:
 - a. Short Pins 1 and 8 together.
 - b. Short Pins 6 and 7 together.
 - c. Attach the Multimeter between Pin 1 (+) and Pin 7 (-). See Illustration 4-10.

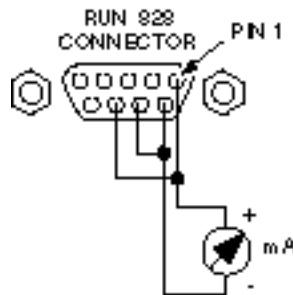

WIRING FOR SENSOR CALIBRATION
ILLUSTRATION 4-10


3. Plug the Magnet Monitor's power cord into an outlet. Turn on the Magnet Monitor.
4. Wait approximately two minutes to allow the system to settle.
5. Press the [Sample] button.
6. Verify the Multimeter reads **75.0 mA**.
7. Turn off the Magnet Monitor's power switch.

8. On the connector of Run 828: Replace the Multimeter with a 686 ohm resistor capable of dissipating four watts, or use the Helium Resistance Box (46-265286G1), to simulate 686 ohms). The connector pins should now be wired as shown in Illustration 4-11.

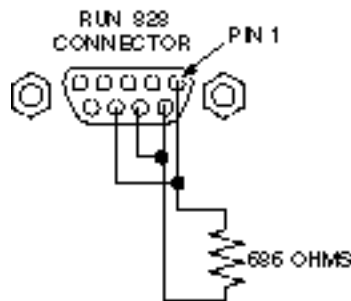

WIRING FOR CURRENT SOURCE VOLTAGE CALIBRATION
ILLUSTRATION 4-11

9. Connect the Multimeter between TP1 on the Magnet Monitor Interface Board and the chassis common.
10. Turn on the Magnet Monitor.
11. Wait approximately two minutes to allow the system to settle.
12. Press the [Sample] button.
13. Adjust the **R7 Gain** potentiometer until the Multimeter reads 5.000 VDC. (See Illustration 4-12 for the location of R7.)
14. Verify the front panel reads 0%, $\pm 1.0\%$.
15. Turn off the Magnet Monitor's power switch.
16. Place a shorting jumper across the 686-ohm test resistor, or set the Helium Resistance Box to zero ohms.
17. Turn on the Magnet Monitor.
18. Wait approximately two minutes to allow the system to settle.
19. Press the [Sample] button.
20. Verify the voltmeter reads 0.000 VDC.
21. Verify the front panel reads 100%, $\pm 1.0\%$.
22. Repeat from Step 12 until no further changes are necessary.
23. Reconnect Run 828 to P403.

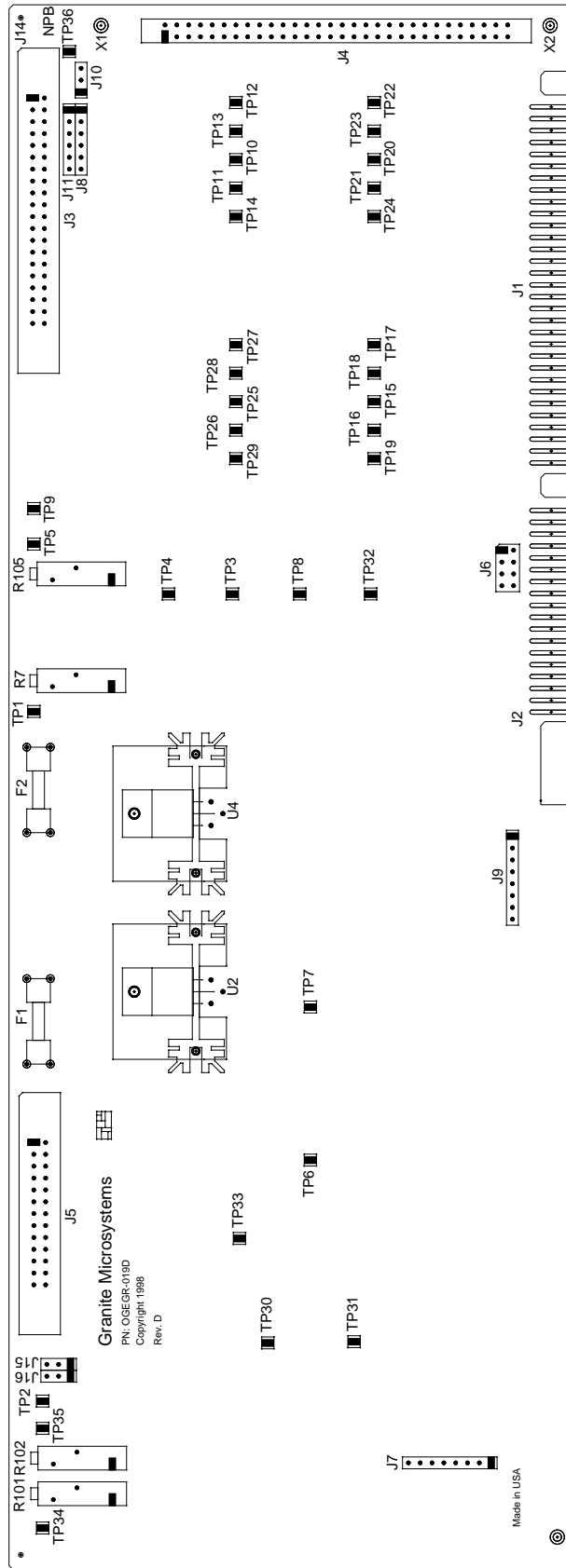

MAGNET MONITOR INTERFACE BOARD
ILLUSTRATION 4-12

4-5-1-2 Checking Helium Calibration of Advantech Magnet Monitor

1. Obtain the Helium Meter Calibration Box (46-306864G1).
2. Observe the helium readings for the vessel.
3. Disconnect the helium level cable at the magnet. These cables are on the Magnet Interconnect, located on the left side of the magnet when viewing from the front.
4. Connect the Helium Meter Calibration Box to the Cable that was going to P302 - Upper. Adjust the resistance on the Helium Meter Calibration Box to the values on the Table 4-1. Press the [Sample] button on the Magnet Monitor. Verify that the helium reading for the "Upper" vessel is within the specification mentioned below.

TABLE 4-1
RESISTANCE/HELIUM

Resistance Value	Nominal Value	Specification
Zero	100%	100% to 98%
230 ohms	66.4	63.4 to 69.4 ohms
460 ohms	32.7	29.7 to 35.7 ohms
686 ohms	0.0	0.0% to 3%

Note

There are no adjustments for helium level calibration in the Advantech Magnet Monitor. If the helium levels are out of range the magnet monitor should be replaced.


4-5-2 Connecting Magnet Monitor to System Cabinet

Notes

For Excite systems, the cable internal to the System's Cabinet is already installed; cable X (no number) does not exist.


Refer to Illustration 8-1, System Interconnect Diagram – LCC/Sumitomo for a diagram of the cable connections described in this section for LX systems.

1. Run 823:
 - a. Connect the MSM1-A1-J10 end to the Magnet Monitor.
 - b. Route the cable to the interface panel at the lower rear of the System Cabinet.
 - c. Install a pair of Connector Joiners (46-265067P1) in an empty 15-Pin opening (J24 if available) on the interface panel.
 - d. Connect the MR2-A11-J24 end of Run 823 to the above.
2. Run X (no number):
 - a. Connect the MR2-A11-J24 end to the interface panel connectors installed above.
 - b. Carefully thread the MR2-IPG-J9 end through to the front of the System Cabinet. and connect to J9 (RF UNBLANK1) of the IPG Board on the front of the cabinet (See Illustration 4-13):

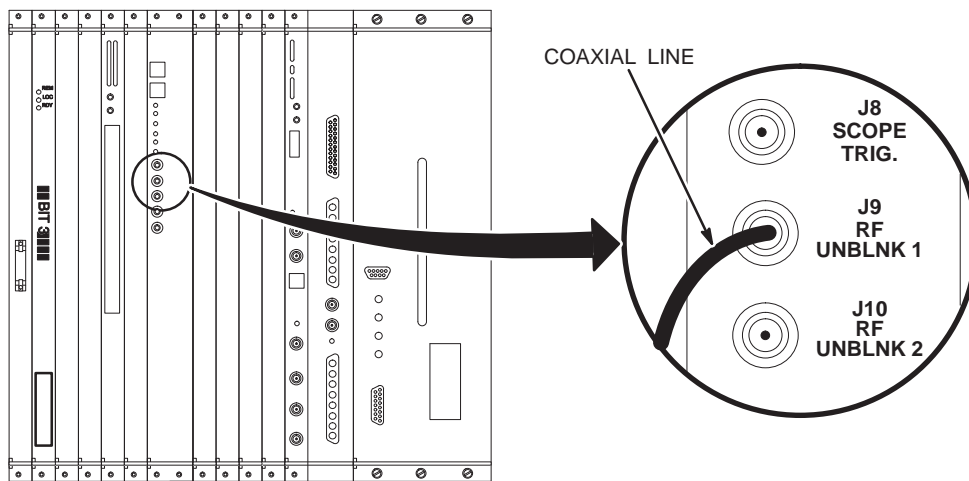

RF UNBLANK CONNECTION
ILLUSTRATION 4-13

- c. At MR2-PDU-J5 in the System Cabinet, remove the existing cable and replace it with a BNC Tee. Connect the existing cable and the MR2-PDU-J5 end of Run X to the BNC Tee. (See Illustration 4-14.)

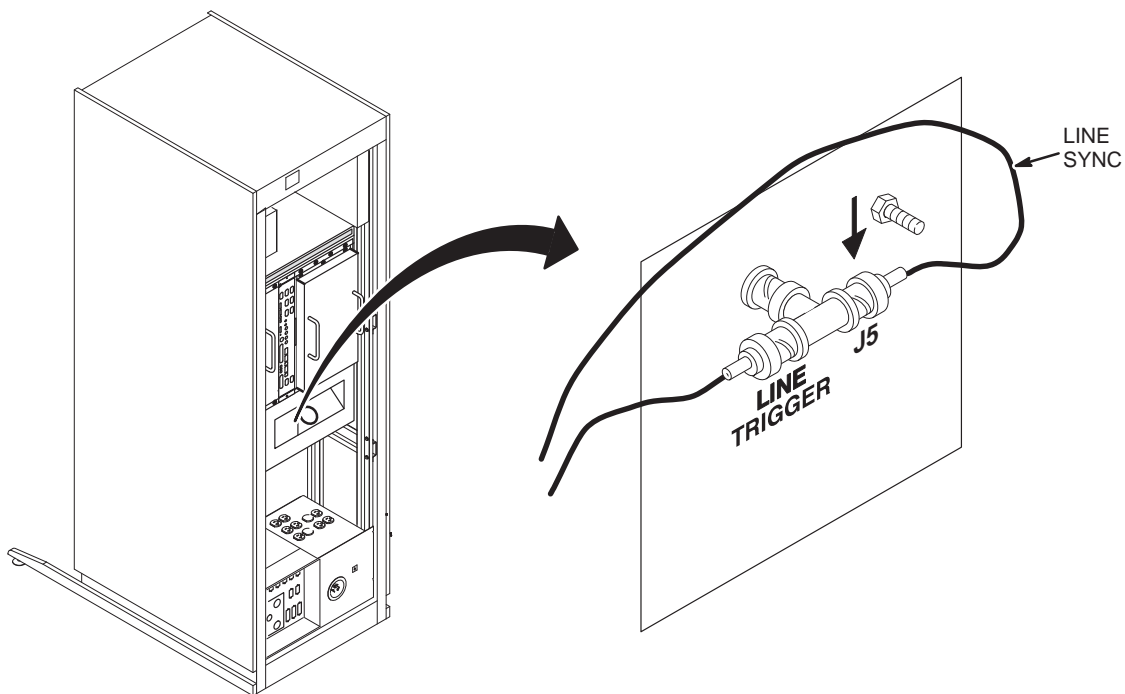

LINE SYNC CONNECTION
ILLUSTRATION 4-14

- 3. Install the Jumper Plug (2239626) on the back of the TYME board in position J5. This jumper eliminates the “low helium” message that the operator would see on the console. Some of the early sites may not have the jumper plug. Pins 4 and 6 of connector J5 (cryo-mon) on the TYME board can be shorted to eliminate the message.

4-6 Installing Global Modem (Advantech Magnet Monitor Only)

1. Place the Global Modem on the top of the Magnet Monitor. Use existing Velcro to secure the modem to the Magnet Monitor.
2. Plug the Power cord into the UPS into the AC receptacle labeled *Magnet Monitor*.
3. Connect cable Run 940, which is provided with the cable kit, between Magnet Monitor J1 and Modem serial connector.

4-7 Installing MUX Box (Optional)

1. Position the MUX box on top of the magnet monitor.
2. Connect the power cord for the MUX box to the MDP into the AC outlet labeled *UPS power*.
3. Connect the customer phone line input (through the UPS) to the MUX box port labeled *phone line in*. Connect the phone line between the Global Modem, and the MUX port 1.

IMPORTANT

Make sure that the MUX box, Port 1 is used for the Magnet Monitor. The MUX box is not connected to the UPS. If the site were to loose power, the MUX box will default to Port 1. This will allow the Magnet Monitor to dial out if the MUX box loses power.

4-8 Installing Remote Alarm (Optional for Advantech Only)

IMPORTANT

The Remote Alarm option will only work on the Advantech Magnet Monitor. It is not compatible with the Granite Magnet Monitor.

1. Place the Remote Alarm near the Operator's Work Station.
2. Remove cable Run 831 from the Magnet Monitor and Run 824.
3. Install the Octopus cable Run 916 into the Magnet Monitor's Connector J7.
4. Run cable 915 between the Magnet Monitor and the Remote Alarm, and make the connections to Magnet Monitor FJ6 and the Remote Alarm Box J1.
5. Connect Run 824 to FJ1 of the Octopus cable Run 916.
6. If planning to connect the Remote Alarm Box to the site's monitoring station, refer to Appendix A.


5 - SOFTWARE CONFIGURATION

1. Connect a serial interface cable between J2 (laptop) on the Magnet Monitor and the serial port on a laptop computer.
2. On the laptop, in Windows XP:
 - a. Open HyperTerminal by selecting: Start > All Programs > Accessories > Communications > HyperTerminal.
 - b. Select a name for magnet monitor connection, such as **Mag Mon**.
 - c. Press **[OK]**.




CONNECTION DESCRIPTION SCREEN
ILLUSTRATION 5-1

- 3. Refer to Illustration 5-2 and select:
 - a. Connect using: **Com1**.
 - b. **[OK]**.

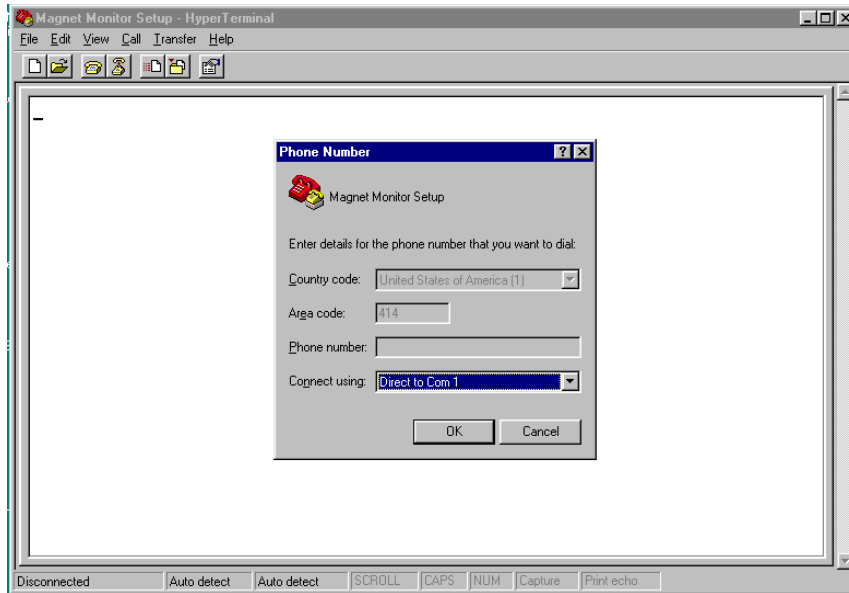

PHONE NUMBER SCREEN
ILLUSTRATION 5-2

- 4. Refer to Illustration 5-3 and select:
 - a. Bits per second: **115200**
 - b. Data bits: **8**

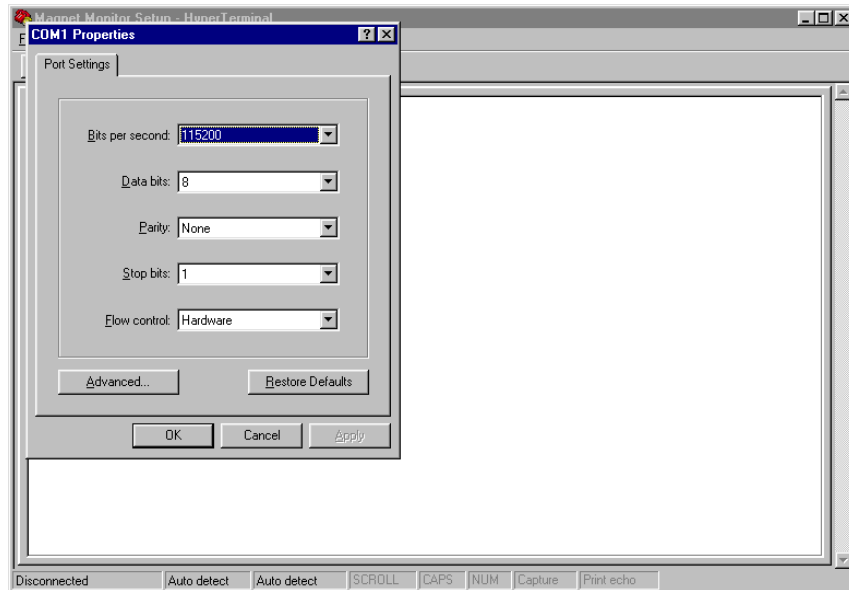

COM1 PROPERTIES SCREEN
ILLUSTRATION 5-3

- 5. Select the *Call* icon to display a login screen.
- 6. Enter the following:
 - a. Login name: **setup**

- b. Password: **monitor**
7. When this message displays on the screen: *Do you wish to change the Configuration*, type [y/n] **Y**.
8. There are two choices for Configuration Type: 1) HFO and 2) LCC. Enter number 1, 2 or Q to quit. Example: **2 [Enter]**.

If necessary, the Compressor Reset Values can be changed. The values below should **not** be changed:

- Do you wish to change the Compressor Configuration values? [y/n] **Y**.
- Do you wish to Enable the Compressor Reset Function? [y/n] **Y**.
- Enter value for Startup Delay Time (minutes) [1:5] **5**.
- Enter value for Reset Delay Time (minutes) [1:30] **10**.
- Enter value for Recheck Delay Time (hours) [1:5] **1**.
- Enter value for Retry Count (count) [0:10] **2**.
- Will this Magnet Monitor be serially connected to a UPS [y/n]?: Select the proper option for this installation.

The Magnet Monitor will now reset.

6 - FUNCTIONAL CHECKS

6-1 Magnet Monitor

1. Press [Sample] button.
2. Check Helium Level for value less than 99%.
3. Check Pressure for a reading that is similar to the gauge reading at the magnet.
4. Allow three days for the magnet to stabilize; then, verify that the magnet pressure is 3.8 to 4.2 psig as indicated at the Magnet Monitor.
5. The following will describe the sequential events that will occur as a magnet monitor is rebooted or powered up. The times mentioned may not be exact.
 - 10 sec. The HDD light located on the left side of the unit will come on as the hard drive is accessed. It will flicker during the remainder of the boot-up cycle.
 - 45 sec. The display will activate with **8.8.8.8**. and all LEDs will illuminate.
 - 60 sec. The helium reading will activate and display the helium level in the magnet vessel. The display cycle between helium level and vessel pressure will begin.

Symptom	Suggested Actions
Message ER1 is on the display	Either there is an open circuit in the helium cable between the Magnet Monitor and the magnet, or the Magnet Monitor is not sending the proper current to the magnet for an accurate helium reading. Check cables between magnet and Magnet Monitor. If continuity is OK, check for 75 mA coming out of the magnet monitor.
Message ER2 is on the seven segment display	Magnet heater is stuck on, possibly due to a software hang or other circumstance. Latched pressure was detected and a reboot of the magnet monitor has already occurred. If problem occurs again, the Magnet Monitor must be replaced.
Message ER3 is on the seven segment display	Heater timeout - Magnet heater was being held on for an extended period. Possible cause might be a slow leak in the magnet, and vessel pressure would never reach the upper limit. In this case, the heater would turn off to conserve helium.

6-2 Remote Alarm Check

The Remote alarm that is located by the Operators Console will activate under the following circumstances:

- Magnet helium level is approaching a critical level, the top vessel is less than 50% or the lower vessel is less than 95%.
- Coldhead temperatures are above specification for greater than 90 minutes.
- Magnet Monitor software is hung, which will prohibit accurate readings.

6-2-1 Functionality of Remote Alarm Box

Verify that magnet pressure is less than 4.1psi. Turn the Cryocooler Compressor off. (After 90 minutes, the Remote Alarm should activate by producing an audible sound and flashing LED.)

7 - Maintenance and Replacement

7-1 Maintenance

The only part of the unit that requires maintenance is the fan filter. This filter must be removed and cleaned with a mild soap and water solution, rinsed in clean water, and reinstalled once per month.

7-2 Magnet Monitor Replacement


There are no Field Replaceable parts within the magnet monitor. It should be replaced as a unit upon failure. Table 7-1 lists some of the FRUs that can be ordered.

TABLE 7-1
MAGNET MONITOR

Item	Description	Part Number	FRU Code	Quantity
1	Advantech Magnet Monitor	2219400	1	1
2	Global Modem	2245794	1	1
3	Cable between Magnet Monitor and Modem	2276568	1	1
4	UPS for Magnet Monitor	2276094	1	1

8 - SCHEMATICS AND INTERCONNECTS

8-1 System Interconnect Diagram for Core Product with Water Flow/Temp Sensor

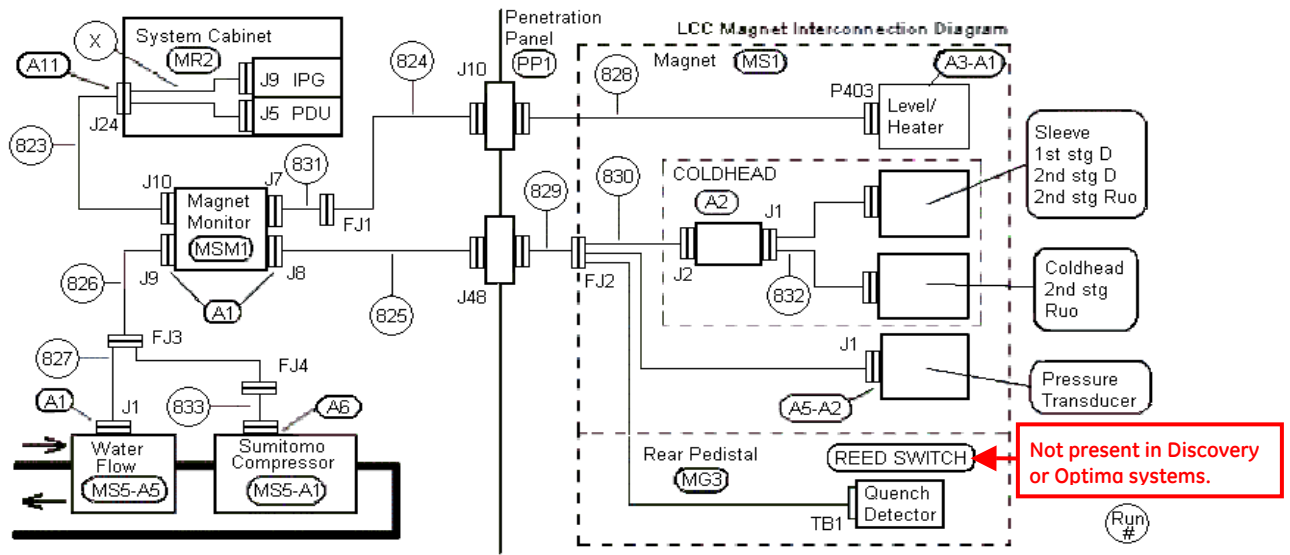

SYSTEM INTERCONNECT DIAGRAM – LCC/SUMITOMO
ILLUSTRATION 8-1


TABLE 8-1
LCC INSTALLED BASE CABLE KIT

Run #	Description	Part Number	From	To
823	Monitor to System Cabinet I/F	46-271603G29	MSM1-A1-J10	MR2-A11-J24
824	Monitor Sensor I/F to Penetration Panel	46-271609G29	PP1-J10	FJ1
825	Monitor Magnet Connector to Penetration Panel	46-271600G29	PP1-J48	MSM1-A1-J8
826	Monitor to Compressor I/F	46-271603G29	MSM1-A1-J9	FJ3
827	Compressor I/F	46-328578P1	FJ3	MS5-A5-A1-J1 and FJ4
828	Penetration Panel to Sensor	46-271609G29	PP1-J10	MS1-A3-A1-P403
829	Penetration Panel to Magnet I/F	46-271601G29	PP1-J48	MS1-FJ2
830	Magnet I/F	2222797 5352964**	MS1-FJ2	MS1-A2-J2, MS1-A5-A2-J1 and MG3-TB1
831	Monitor to Sensor I/F	2204488-5	FJ1	MSM1-A1-J7
832	Buffer Amp to Cooling Sleeves	2214062	MS1-A2-J1	MS1-Sleeve and MS1-Coldhead
833*	Compressor to Compressor I/F	2225805	FJ4	MS5-A1-A6-JR
X (No #)	System Cabinet I/F	2204485	MR2-A11-J24	MR2-IPG-J9 and MR2-PDU-J5

* Run 833 is for a Sumitomo compressor. This run is compressor specific.

** Discovery and Optima only.

8-2 System Interconnect Diagram for S-Series and CX Magnets

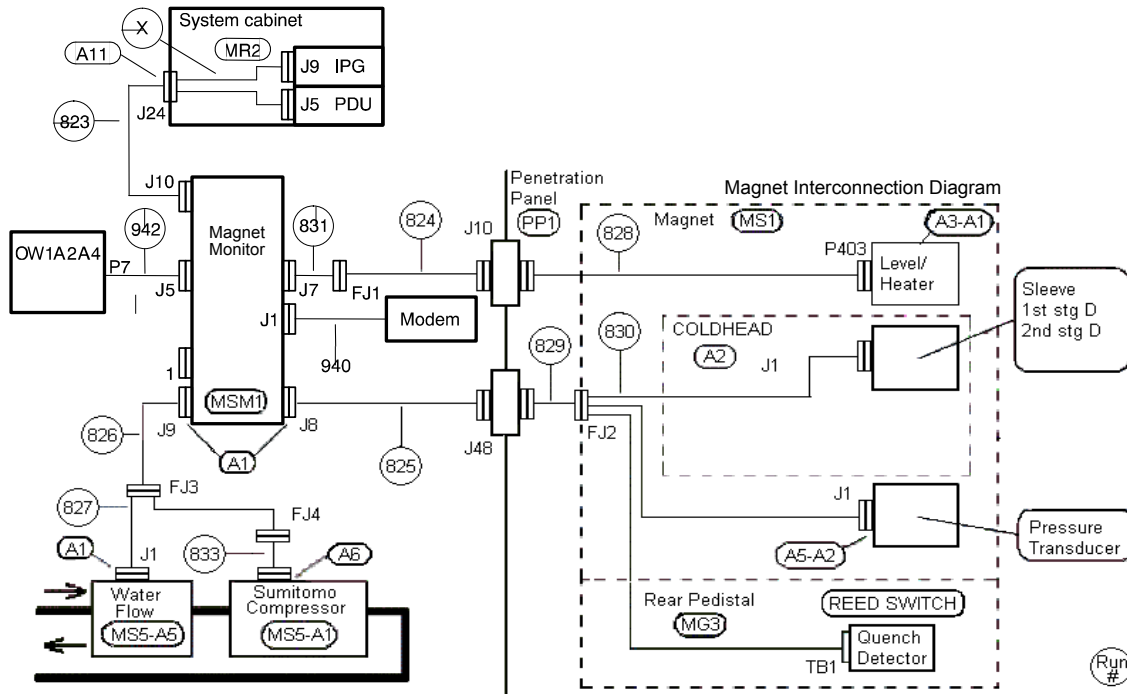


SYSTEM INTERCONNECT DIAGRAM – S-SERIES AND CX MAGNET
ILLUSTRATION 8-2

IMPORTANT

Do NOT use cable Run 830 included in the Magnet Monitoring Cable Kit. Verify that the cable with the 9-pin sub-D connector is used for routing to the Coldhead Sleeve.

8-3 System Interconnect Diagram for UPS (Option for Twin)

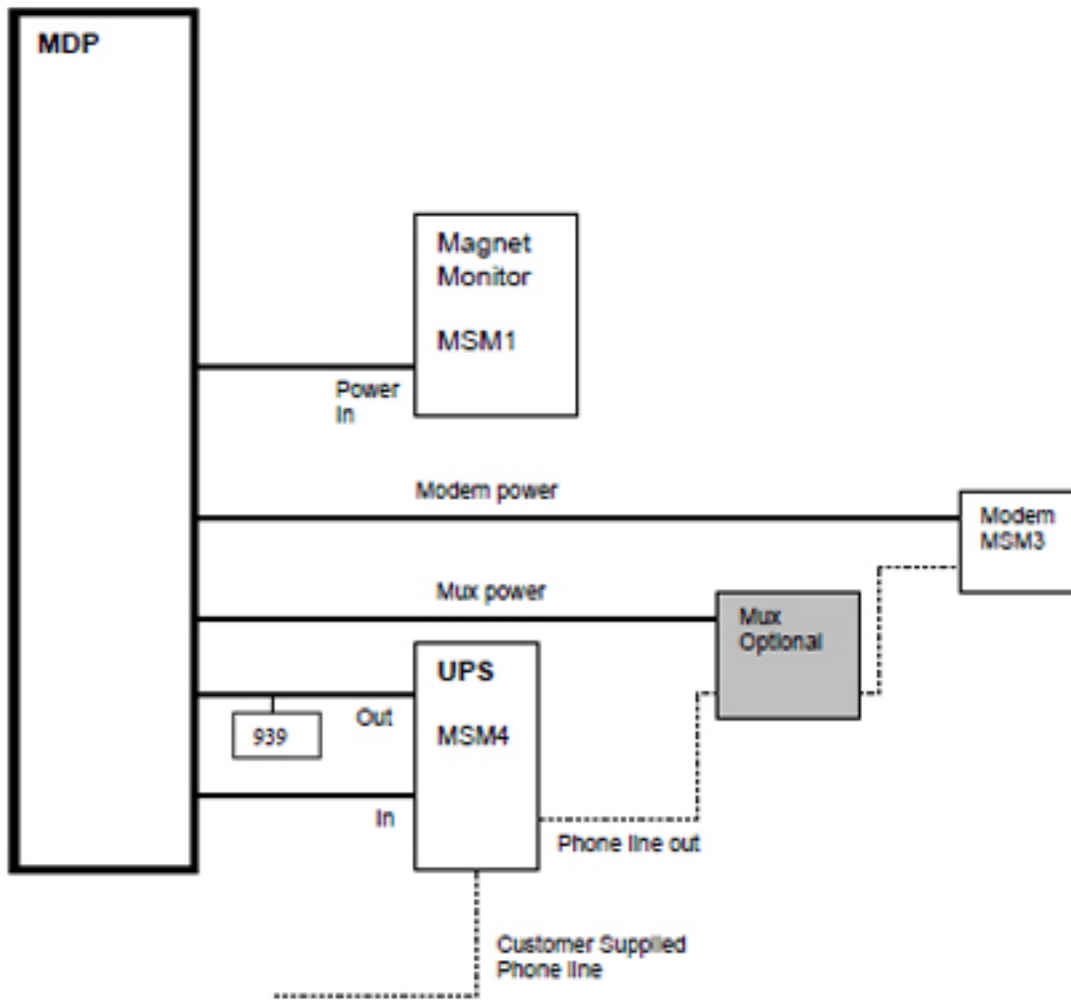

INNERCONNECT DIAGRAM FOR UPS POWER AND PHONE LINE
ILLUSTRATION 8-3

TABLE 8-2
POWER/PHONE CABLE LIST

Run #	Description	Part Number	From	To
939	MDP to UPS for Power Out (included with UPS)	2276569	UPS Out	MDP In
N/A	MDP to UPS (supplied with UPS)	N/A	MSM4	MDP
N/A	Monitor to MDP (supplied with Monitor)	N/A	MSM1 Power	MDP
N/A	Modem to MDP (supplied with Modem)	N/A	MSM3	MDP

All internal Magnet Monitor cables are straight-through cables with one-to-one correspondence between pins for each connector, except for the numbered cables. Their pin-outs are shown in the following tables:

TABLE 8-3
INTERNAL OCTOPUS CABLE

Header Pin No. (J4-)	DB Pin Number	DB Connector
1	1	Cryogen Level J7 (DB25 Female)
2	14	Cryogen Level J7 (DB25 Female)
3	2	Cryogen Level J7 (DB25 Female)
4	15	Cryogen Level J7 (DB25 Female)
5	3	Cryogen Level J7 (DB25 Female)
6	16	Cryogen Level J7 (DB25 Female)
7	4	Cryogen Level J7 (DB25 Female)
8	17	Cryogen Level J7 (DB25 Female)
9	5	Cryogen Level J7 (DB25 Female)
10	18	Cryogen Level J7 (DB25 Female)
11	6	Cryogen Level J7 (DB25 Female)
12	19	Cryogen Level J7 (DB25 Female)
13	7	Cryogen Level J7 (DB25 Female)
14	20	Cryogen Level J7 (DB25 Female)
15	8	Cryogen Level J7 (DB25 Female)
16	13	Scan Room J8 (DB25 Male)
17	25	Scan Room J8 (DB25 Male)
18	12	Scan Room J8 (DB25 Male)
19	24	Scan Room J8 (DB25 Male)
20	11	Scan Room J8 (DB25 Male)
21	23	Scan Room J8 (DB25 Male)
22	10	Scan Room J8 (DB25 Male)
23	22	Scan Room J8 (DB25 Male)
24	9	Scan Room J8 (DB25 Male)
25	21	Scan Room J8 (DB25 Male)
26	8	Scan Room J8 (DB25 Male)
27	20	Scan Room J8 (DB25 Male)
28	7	Scan Room J8 (DB25 Male)
29	19	Scan Room J8 (DB25 Male)
30	6	Scan Room J8 (DB25 Male)
31	18	Scan Room J8 (DB25 Male)
32	5	Scan Room J8 (DB25 Male)
33	17	Scan Room J8 (DB25 Male)
34	4	Scan Room J8 (DB25 Male)
35	16	Scan Room J8 (DB25 Male)
36	3	Scan Room J8 (DB25 Male)
37	15	Scan Room J8 (DB25 Male)
38	2	Scan Room J8 (DB25 Male)
39	14	Scan Room J8 (DB25 Male)
40	1	Scan Room J8 (DB25 Male)
41	1	Compressor J9 (DB15 Female)
42	9	Compressor J9 (DB15 Female)
43	2	Compressor J9 (DB15 Female)

Header Pin No. (J4-)	DB Pin Number	DB Connector
44	10	Compressor J9 (DB15 Female)
45	3	Compressor J9 (DB15 Female)
46	11	Compressor J9 (DB15 Female)
47	4	Compressor J9 (DB15 Female)
48	12	Compressor J9 (DB15 Female)
49	5	Compressor J9 (DB15 Female)
50	13	Compressor J9 (DB15 Female)
51	6	Compressor J9 (DB15 Female)
52	14	Compressor J9 (DB15 Female)
53	7	Compressor J9 (DB15 Female)
54	15	Compressor J9 (DB15 Female)
55	8	Compressor J9 (DB15 Female)
56	7	System Cabinet J10 (DB15 Male)
57	6	System Cabinet J10 (DB15 Male)
58	11	System Cabinet J10 (DB15 Male)
59	10	System Cabinet J10 (DB15 Male)
60	9	System Cabinet J10 (DB15 Male)

TABLE 8-4
INTERNAL INTERFACE BOARD TO ADC BOARD CABLE

IDC 2x20 Header (Interface Board End)	IDC Female DB-37 (ADC Board End)
1	1
2	20
3	2
4	21
5	3
6	22
7	4
8	23
9	5
10	24
11	6
12	25
13	7
14	26
15	8
16	27
17	9
18	28
19	10
20	29
21	11
22	30
23	12
24	31
25	13
26	32

IDC 2x20 Header (Interface Board End)	IDC Female DB-37 (ADC Board End)
27	14
28	33
29	15
30	34
31	16
32	35
33	17
34	36
35	18
36	37
37	19
38	No Connection
39	No Connection
40	No Connection

9 - GLOSSARY

Cryogen

A coolant in the form of a liquefied gas such as helium or nitrogen used to cool the magnet to superconducting temperatures.

Excite

Signa LX system with 8-channel technology, released in Fall of 2002.

Gauss

Unit of measurement for the strength of a magnetic field.

Generic Back Plane

A circuit board with only interconnection hardware (such as an ISA bus), but no electronic circuitry.

Industry Standard Architecture (ISA) Bus

A series of common connection points used to connect ISA compatible boards.

LCC

Low-cost cryogen magnet.

Magnet Interface Board

Provides a connection between the Magnet Monitor and the magnet and associated support hardware and electronics such as compressors and cooling shields. The board also drives the display board and conditions analog signals before sending them to the analog to digital converter board (ADC).

Modem

A device used to allow computers to communicate over telephone lines.

Network Interface Card (NIC)

Provides circuitry for connecting a computer to a network.

Penetration Panel

A connection panel used to isolate the Magnet Room's electrical cables from external cables.

Single Board Computer (SBC)

A single board that contains the main computing hardware such as the central processing unit (CPU), memory, controllers for the disk drives, etc. This board differs from computer "mother boards" in that it does not have a built-in bus for connecting other boards.

TGWC (Twin Gradient Water Cooler)

A water chiller used to cool the gradient coil. This system provides data to the Magnet Monitor. It is used when upgrading an existing magnet to the Twin product.

TSCC (Twin Systems Cooling Cabinet)

A water chiller used to cool the cryo-cooler compressor and the gradient coil. This system provides data to the Magnet Monitor.

Twin

MR system that contains a dual gradient coil. This system uses a vacuum between the magnet and enclosure/gradient coil to reduce noise.

UPS

Uninterruptable Power Supply that powers the magnet monitor upon power outage.

APPENDIX A – CONNECTING REMOTE ALARM BOX TO HOSPITAL

The Remote Alarm Box can be connected to the hospital's warning system. This will inform the security staff of an issue when the MR suite is vacant. The Remote Alarm will activate for a 'low helium' alarm or if the compressor is off for more than 10 minutes.

To connect the Remote Alarm Box to a remote panel, connect Pins 4 and 9 to the device. The relay in the Remote Alarm Box is normally open, but will close when the alarm is activated.


APPENDIX B – CONNECTING PRESSURE TRANSDUCER TO MAGNET



The Pressure Transducer must be installed by personnel, who are trained in Magnet Safety per 2301164PRE. Do NOT attempt to install pressure transducer if not trained in Magnet Safety.

B-1 Connecting Pressure Transducer on S-II, S-III, S-III Magnashield and 0.5T MAX Magnets

1. Illustration B-1 shows how to connect the Pressure Transducer on S-II, S-III, S-III Magnashield, and 0.5T MAX (without alternate venting) magnets.

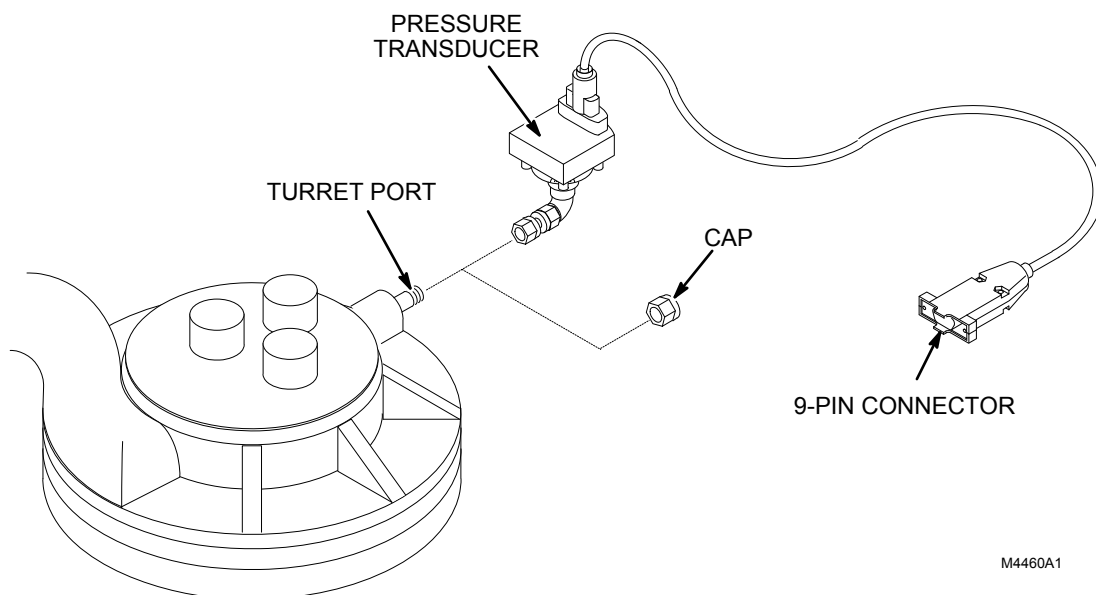

S-II, S-III, S-III MAGNASHIELD, AND 0.5T MR MAX (WITHOUT ALTERNATE VENTING) PRESSURE TRANSDUCER CONNECTIONS
ILLUSTRATION B-1

2. Using a nonmagnetic 9/16 in. combination wrench, remove the Swagelok cap from the turret port quickly so that ice does not form on the port. (If a plastic line is connected to the turret port, disconnect it. Leave the plastic line on top of the magnet.)




Always use two wrenches when installing the pressure transducer onto the magnet turret port. Hold the turret port with one wrench to prevent it from further tightening into the plenum while tightening the pressure transducer. The plenum can easily crack and will require ramping the magnet down to replace it.

9. Install the Pressure Transducer (46-320840P1) onto the turret port, and finger tighten until Pressure Transducer cannot be rotated. Tilt the Pressure Transducer up to prevent condensation from collecting in the Transducer.
10. Using the nonmagnetic adjustable and 9/16 in. wrenches, tighten this connection 1/4 turn past finger tight.
11. Using leak detection fluid (Snoop), and check for leaks in the fittings.
6. Route the 9-Pin Connector, attached to the Pressure Transducer, toward the rear of the magnet.

B-2 Connecting Pressure Transducer on S-IV, S-V, S-X, and S-XC Magnets

1. Illustration B-2 shows how to connect the Union Tee and the Pressure Transducer on S-IV, S-V, S-X, and S-XC magnets.

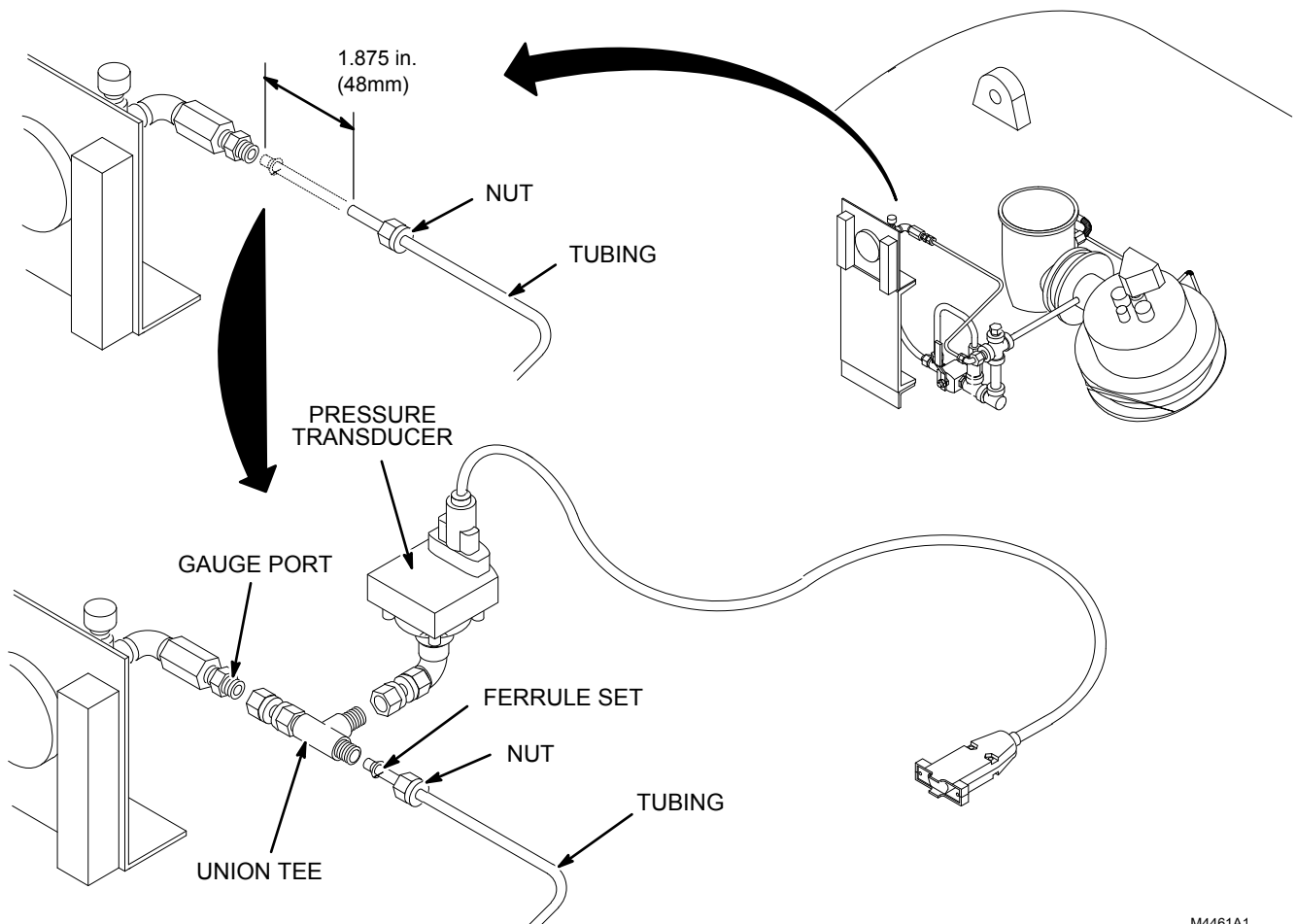

M4461A1

**S-IV, S-V, S-X, AND S-XC UNION TEE AND PRESSURE TRANSDUCER CONNECTIONS
ILLUSTRATION B-2**

2. Loosen the nuts on each end of the 1/4 in. copper tubing connected to the instrumentation lead, and remove the tubing.

3. Since the Union Tee (46-252294P1) will be connected to this copper tubing, remove enough of the tubing to make room for the Tee. Line up the Union Tee with the copper tube. Mark where to cut the tubing, approximately 1-7/8 inches (48 mm) from the end.
4. Take the tubing outside of the Magnet Room, and use the tubing cutter to cut the tubing at the mark. Retain the nut for use during installation.


Note

Space restrictions may require rotation of the Union Tee 180 degrees and installation of the Pressure Transducer on the other side of the tubing.

5. Connect the Union Tee to the gauge port and finger tighten. The tee should be aligned as shown in Illustration B-2. Use the nonmagnetic 9/16 in. combination wrench to tighten this connection 1/4 turn past finger tight.
6. Reinstall the uncut end of the tubing to the location from which it was removed (in Step 2) and finger tighten.
7. Slide the nut (from Step 2) and a ferrule set (from the Union Tee bag in the kit) onto the other end of the tubing.
 - a. Push this end of the tubing into the Union Tee until it bottoms out.
 - b. Slide the nut and ferrule set onto the tee and finger tighten the nut.
 - c. Using the 9/16 in. combination wrench and the adjustable wrench, tighten this connection 1-1/4 turns past finger tight to "swage" the fitting.
8. Install the Pressure Transducer (46-320840P1) onto the Union Tee and finger tighten.
9. Tilt the Pressure Transducer up to prevent condensation from collecting in the Transducer. Use the wrench to tighten this connection 1/4 turn past finger tight.
10. Using leak detection fluid (Snoop), and check for leaks in the fittings.
11. Route the 9-Pin Connector, attached to the Pressure Transducer, to the side of the magnet.

B-3 Connecting Pressure Transducer on CX Magnets

1. Illustration B-3 shows how to connect the Union Tee and the Pressure Transducer on CX magnets.

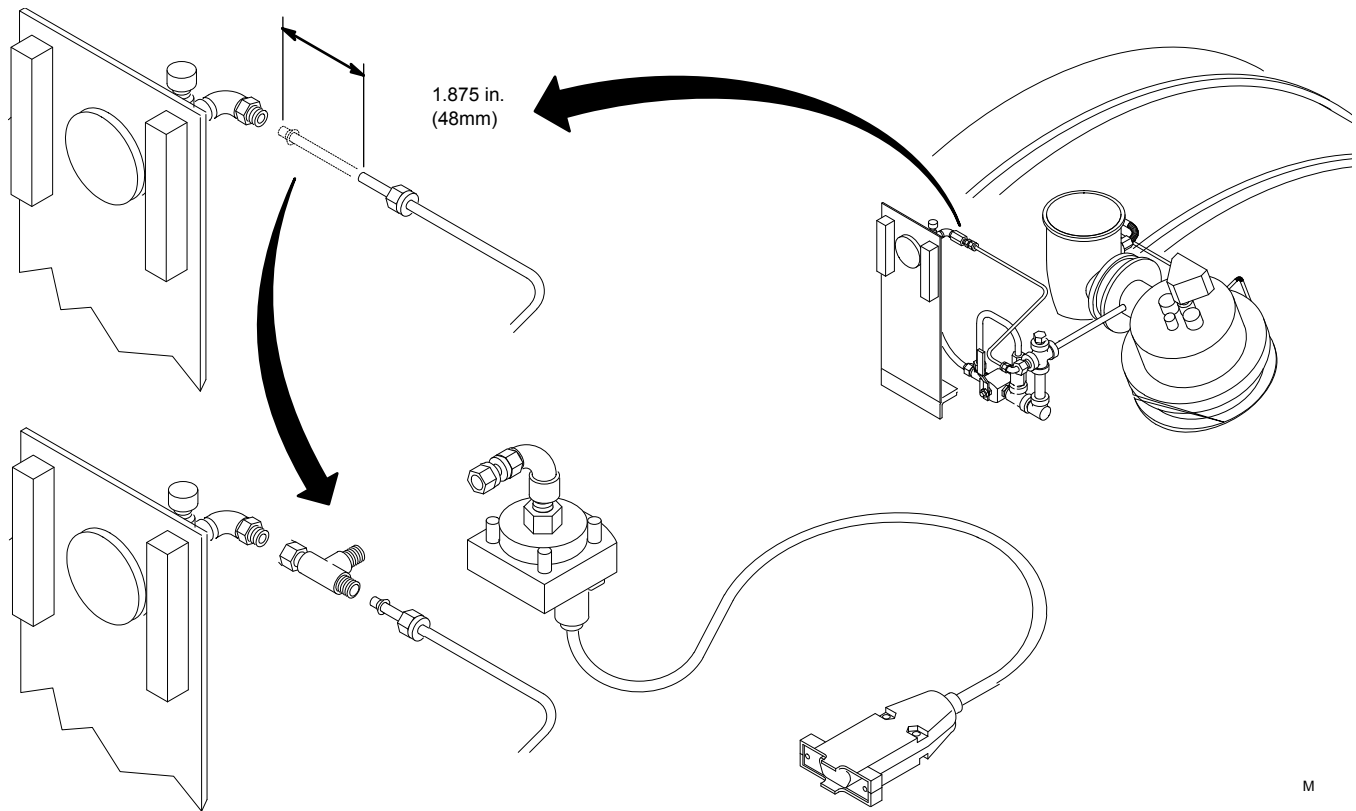

CX UNION TEE AND PRESSURE TRANSDUCER CONNECTIONS
ILLUSTRATION B-3

2. Loosen the nuts on each end of the 1/4 in. copper tubing connected to the instrumentation lead, and remove the tubing.
3. Since the Union Tee (46-252294P1) will be connected to this copper tubing, remove enough of the tubing to make room for the Tee. Line up the Union Tee with the copper tube. Mark where to cut the tubing, approximately 1-7/8 inches (48 mm) from the end.
4. Take the tubing outside of the Magnet Room, and use the tubing cutter to cut the tubing at the mark. Retain the nut for use during installation.

Note

Space restrictions may require rotation of the Union Tee 180 degrees and installation of the Pressure Transducer on the other side of the tubing.

5. Connect the Union Tee to the gauge port and finger tighten. The tee should be aligned as shown in Illustration B-3. Use the nonmagnetic 9/16 in. combination wrench to tighten this connection 1/4 turn past finger tight.
6. Reinstall the uncut end of the tubing to the location from which it was removed (in Step 2) and finger tighten.

7. Slide the nut (from Step 2) and a ferrule set (from the Union Tee bag in the kit) onto the other end of the tubing.
 - a. Push this end of the tubing into the Union Tee until it bottoms out.
 - b. Slide the nut and ferrule set onto the tee and finger tighten the nut.
 - c. Using the 9/16 in. combination wrench and the adjustable wrench, tighten this connection 1-1/4 turns past finger tight to “swage” the fitting.
8. Install the Pressure Transducer (46-320840P1) onto the Union Tee and finger tighten.
9. Tilt the Pressure Transducer up to prevent condensation from collecting in the Transducer. Use the wrench to tighten this connection 1/4 turn past finger tight.
10. Using leak detection fluid (Snoop), and check for leaks in the fittings.
11. Route the 9-Pin Connector, attached to the Pressure Transducer, to the side of the magnet.

REVISION HISTORY

REV	DATE	AUTHOR	PRIMARY REASON FOR CHANGE
0	Jan. 1999	P. Kargard	Initial release.
1	Nov. 1999	P. Kargard	Updates after initial release feedback.
2	Oct. 2001	P. Kargard	Modifications for Advantech and Twin.
3	March 2005	P. Kargard	Modifications for installation on older style magnets with Excite HD upgrade.
4	June 2005	P. Kargard	Modifications for laptop with XP configuration.
5	July 2005	P. Kargard	Removed wording for Windows 98 laptop.
6	Sept. 2005	R. Stetz	Reworked "Graceful Shutdown Procedure".
7	Nov. 2006	K. Keshena	Added Section "Attaching Preamp to Magnet".
8	Sept. 2010	R. David	Added a Note in Illustration 8-1 and Section 4-1-4: Connecting Reed Switch stating that it is not used for Discovery or Optima Systems per ECO 2089099.

www.gehealthcare.com



Imagination at work