

SERVICE NOTES

Nov 19th, 2002

Page 1 of 1

FILE TITLE

Signa Ovation System

SN62717

Problem with 2D FIESTA plus Gating Option

Problem:

The following problem can be occurred if 2D FIESTA plus Gating Option is performed. Fast TOF GRE

- Strong Ghost appears on the Image
- Scan stops since the system recognizes the little disorder of heart pulse as an irregular pulse.
- There is 100~200msec differencial between "Triger delay" displayed on Image and the real "Triger delay".

Solution:

Do not perform the scan with 2D FIESTA plus Gating Option.

From the MFO3 M4 software, this problem will be solved.

Yutaka Masumo

GEYMS TP Gr. -MR