

# **FMI ADMINISTRATIVE INFORMATION FMI No: 67505**

**FMI Title : MAGNET RUBBER PAD REPLACEMENT FOR Signa Ovation**

<b>FMI TYPE</b>	<b>CHARGE CODES</b>	<b>IMPLEMENTATION DATES</b>
<input type="checkbox"/> <b>Mandatory Safety</b> <input checked="" type="checkbox"/> <b>Mandatory</b> <input type="checkbox"/> <b>Mandatory on Request</b>	<b>GEMS-AM:</b> <b>GEMS-E:</b> <b>GEMS-A:</b>	<b>Issued Date: T.B.D</b>  <b>Closed Date: T.B.D</b>

**Staged FMI** **Customer List** provided by FMI Administration. If you identify additional units within effectivity range, but they are not shown on the customer list, contact FMI Administration in your Pole

**Parts** and any special tools are furnished automatically. To address shortages and ordering exceptions, contact FMI Administration in your Pole.

**Parts Ordering:** GEMS-Americas, Contact your Region logistics coordinator  
 GEMS-Europe, Order FMI kits from GPO/EPL Buc  
 GEMS-Asia, Contact your Region logistics coordinator

**NO Parts** are required to complete this FMI.

**Batch with FMI(s) :** \_\_\_\_\_  
 To minimize costs, combine implementation of this FMI with FMI's listed for all customers where the effectivity is the same (Please see your customer lists).

**Non-Staged FMI : No Customer List is provided.** Each service location must identify affected units and place part orders as indicated in the Special Instructions.

## **Technical Support**

<b>GEMS-Americas</b>	<b>GEMS-Europe</b>	<b>GEMS-Asia</b>
GE Medical Systems InSite Center (414) 524-5637 800-321-7937 Milwaukee, WI 53201, USA	OnLine Center Europe Phone: 33 (0) 1 01 30 83 13 00 Fax: 33 (0) 1 01 30 70 99 70	OnLine Center Asia Phone: 81-426-48-2940 Fax: 81-426-48-2905

**Special Instructions:**

**Unit Tracked: See Effectivity**

**FMI Instruction Part No. : 2317560**

**Do NOT leave unsecured on site**



## MAGNET RUBBER PAD REPLACEMENT FOR Signa Ovation

### EFFECTIVITY :

All Signa Ovation Magnet with the following serial numbers:

YC#009, YC#010, YC#011, YC#012, YC#013, YC#015, YC#016, YC#017, YC#018, YC#019

### PURPOSE :

The height of the Magnet listed in effectivity is 5mm higher than normal YC Magnet. Because of this reason, it is necessary to replace the magnet leg rubber pad from 20mm thickness to 15mm thickness. It is necessary to perform this procedure just before placing the Magnet on the floor of Magnet Room.

### FURNISHED MATERIALS :

FMI kit 2318838, consisting of the following:

ITEM	PART NO.	DESCRIPTION	QTY
1	2316892	Foot Rubber	1
2	2317560	FMI Instruction	1

### SPECIAL TOOLS AND TEST EQUIPMENT :


Signa Ovation Magnet Delivery & Installation Manual which is shipped with the Magnet.

**PROCEDURE :**



**Note**

Perform this procedure just before placing the Magnet on the floor of Magnet Room.


1. Jack up the Magnet.

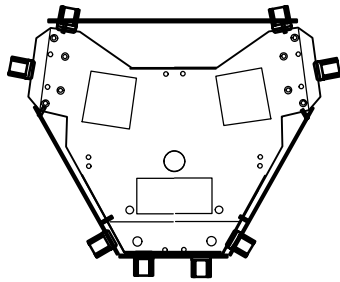


Top View

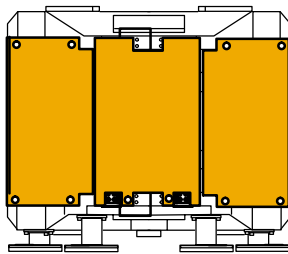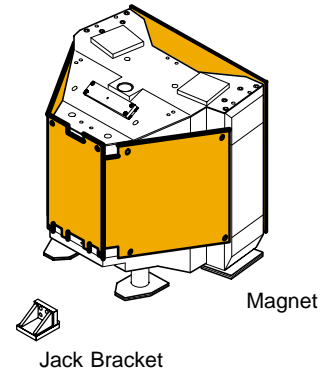

Front View

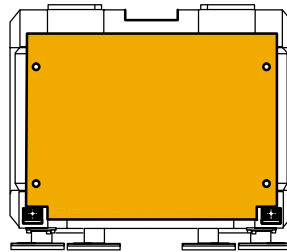

Rear View

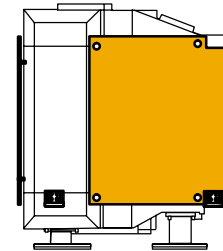


Side View

**JACK UP POSITION  
ILLUSTRATION 1**

**PROCEDURE (continued) :**


- 2. Place the 15mm thickness Magnet Leg Rubber Pad under the Magnet leg.  
Perform this for all regs.

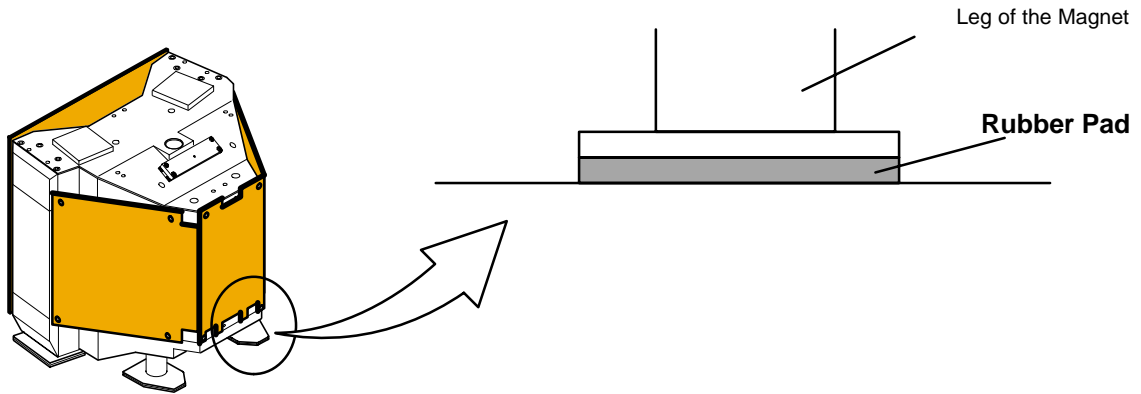

**MAGNET LEG RUBBER PAD REPLACEMENT**  
ILLUSTRATION 2

**Note**

The number “15” is recorded on the each surface of the 15 mm thickness rubber pad. Identify the rubber pad by this.

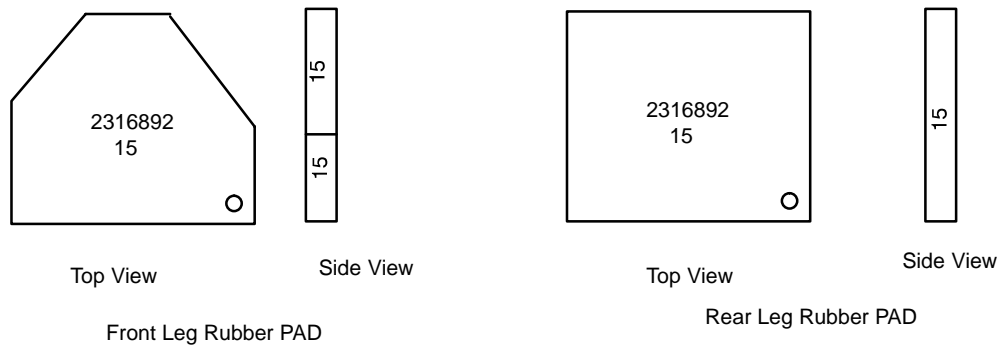

**RUBBER PAD IDENTIFICATION**  
ILLUSTRATION 3