

Rev 0

# TABLE OF CONTENTS

TABLE OF CONTENTS.....	1
1. Overview .....	2
2. Body Coil Registration .....	2

Rev 0

### 1. Overview

This procedure describes how to register Body Coil in Config File.

It is necessary to register the Body Coil in Coil Config File to perform RF Output check, Body Power Check, and LV Shim.


### 2. Body Coil Registration

- 1. Click [Service Desktop]..



**SERVICE DESKTOP ILLUSTRATION 1**

- 2. Select "Config File Manager" and Click [Start....]..

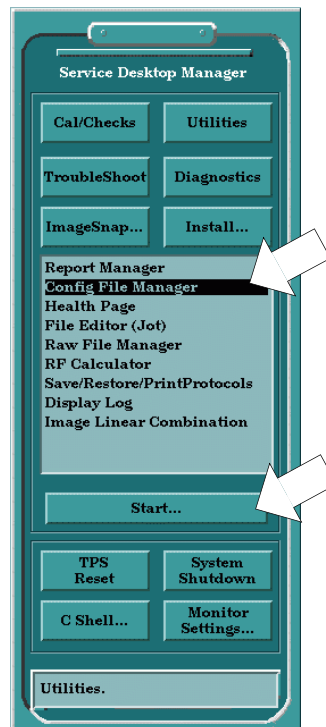


**SERVICE DESKTOP MANGER ILLUSTRATION 2**

Rev 0

## 2. Body Coil Registration (continued)

3. Click [Coil Config File]. .

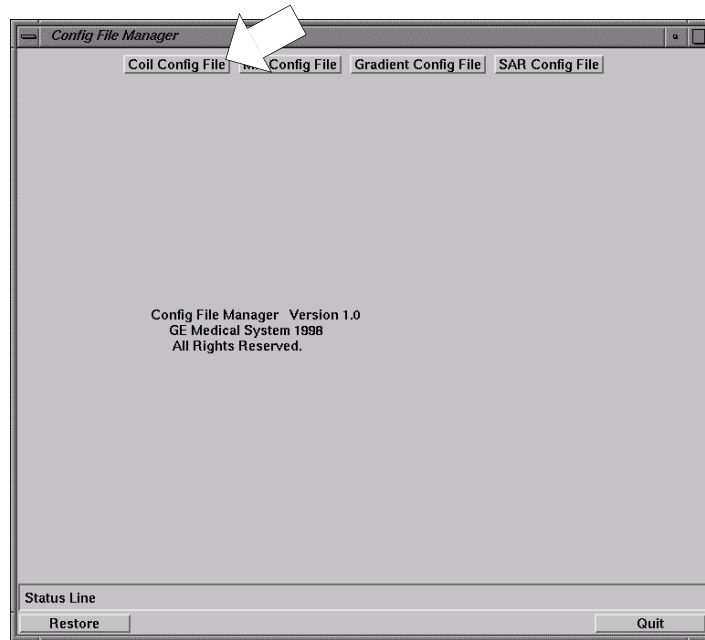


**CONFIG FILE MANAGER**  
ILLUSTRATION 3

4. Select the “**Body**” from “Availavle Coils” and click [ADD] button.

5. Verify that the Coil is added in Site Coil.

6. Click [Quit].

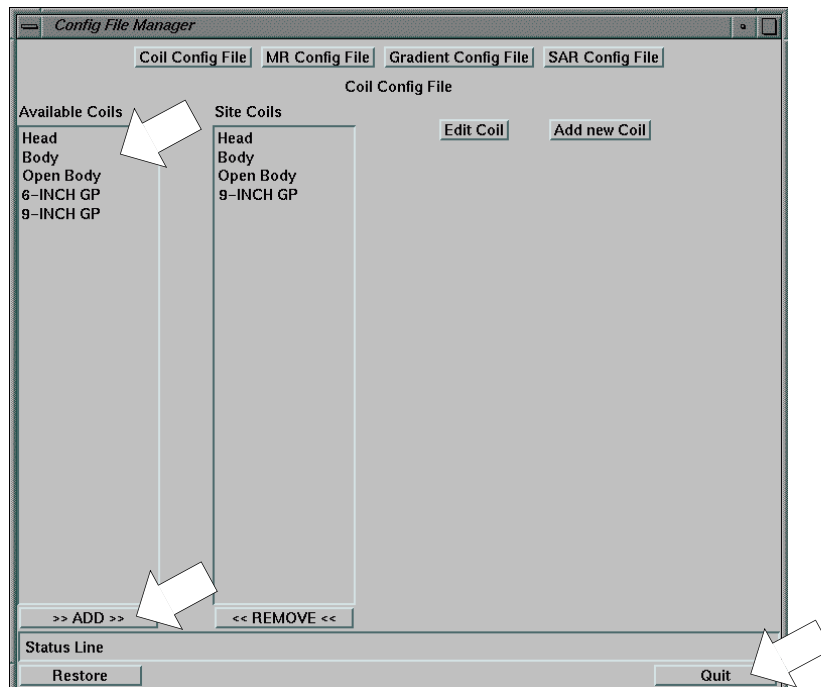

**CONFIG FILE MANAGER**  
ILLUSTRATION 4


Rev 0

**2. Body Coil Registration (continued)**

7. Click [Yes].

**REALLY QUIT  
ILLUSTRATION 5**

8. Click [Save].

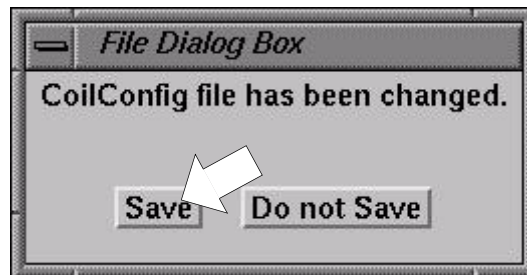**FILE DIALOG BOX  
ILLUSTRATION 6**

9. It is necessary to reboot the system to update the coil config file.

Rev 0

**Revision History**

<b>Rev</b>	<b>Date</b>	<b>Auther</b>	<b>Primary Reasons For Change</b>
0	Feb 9, 2001	Y. Masumo	Initial Release