

TABLE OF CONTENTS

TABLE OF CONTENTS.....	1
1. problem	2
2. Solution	2

Rev 0


1. Problem

Coil config file parameter of Head coil and 9 inch coil contains wrong setting. Because of it, it is necessary to edit the parameter. This problem will be solved from M4 system.

2. Solution

If Head Coil or 9 inch coil is registered, perform the following procedure. Refer to “Coil Config Parameter Setting” manual to open the coil config file manager.

1. Select Head (or 9-inch GP) from site coil.
2. Click [Edit Coil].

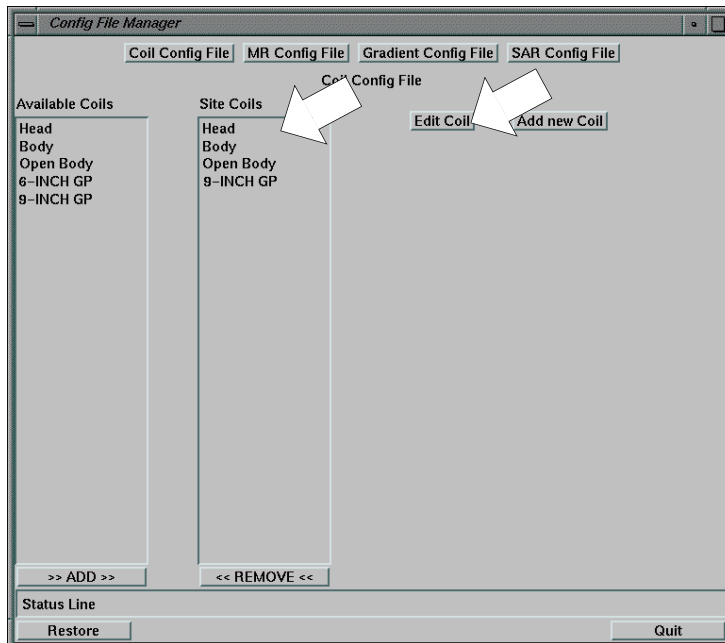

COIL CONFIG MANAGER
ILLUSTRATION 1

Rev 0



2. Solution

3. Edit the following parameter.

•**Head:** Linear(0) vs. Quadrature(1) 1(wrong) => 0(correct)

•**9 inch GP:** Linear(0) vs. Quadrature(1) 0(wrong) => 1(correct)

4. After edit the coil config, click [OK].

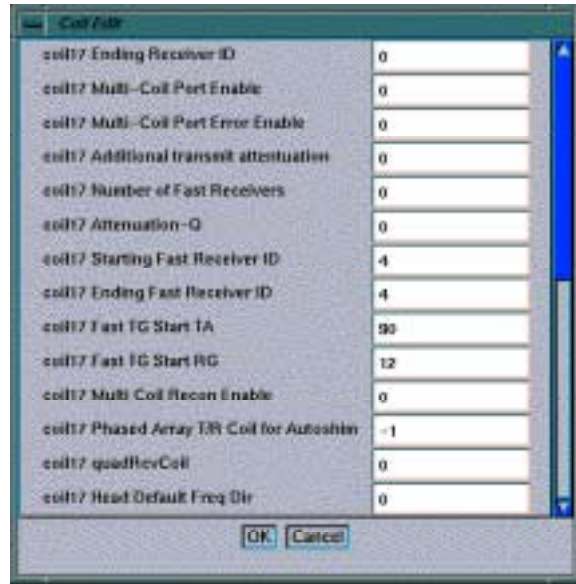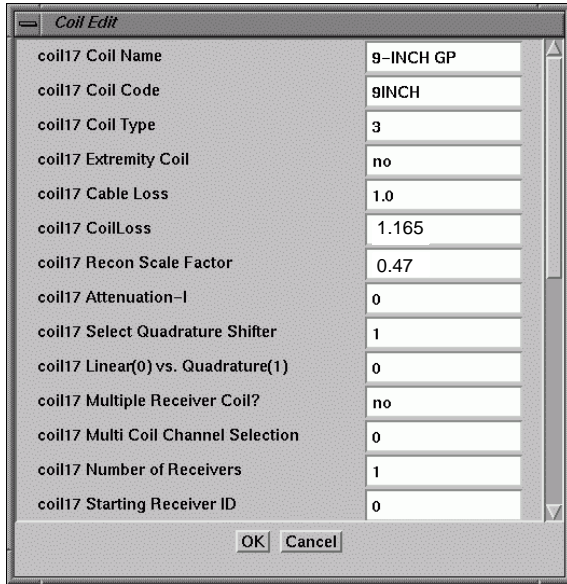


EDIT COIL
ILLUSTRATION 2

Rev 0


2-0 Add Coil Config(If Coil Name is NOT in the Available Coils) (continued)

5. Click [Quit].

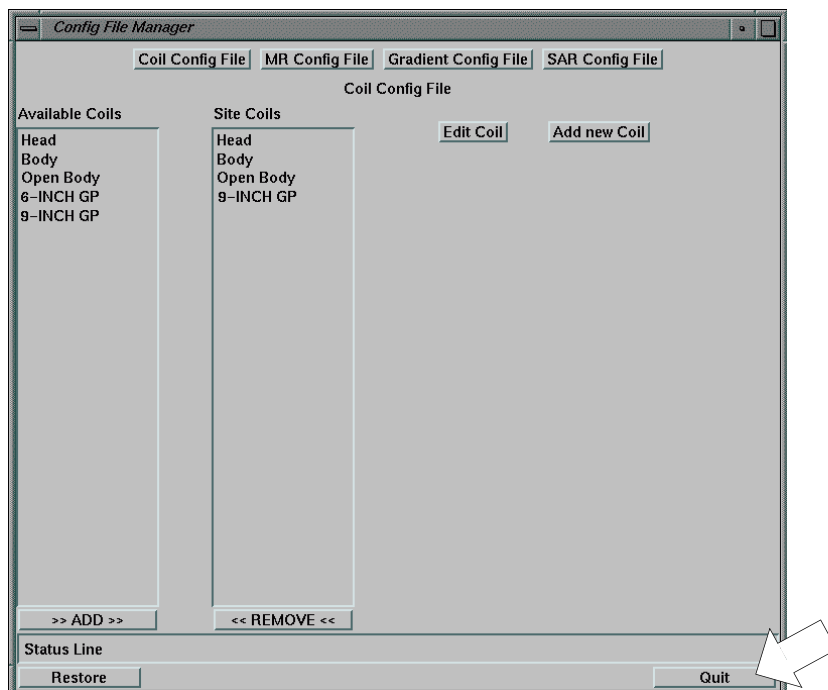

COIL CONFIG MANAGER
ILLUSTRATION 3

6. Click [Yes].

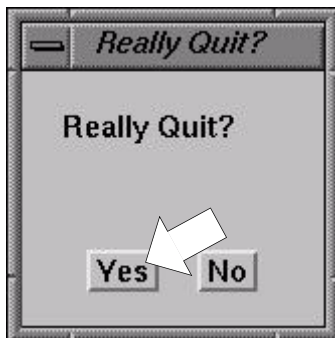

REALLY QUIT
ILLUSTRATION 4

7. Click [Save].



FILE DIALOG BOX
ILLUSTRATION 5

8. It is necessary to reboot the system to update the coil config file.

Rev 0

Revision History

Rev	Date	Auther	Primary Reasons For Change
0	Feb 9, 2001	Y. Masumo	Initial Release