

Rev 0

TABLE OF CONTENTS

TABLE OF CONTENTS	1
1- PREPARATION	2
1-1 OVERVIEW	2
1-2 GND CABLE WITH SCREW KIT (2289497)	2
2- GROUND CABLE CONNECTION	2

Rev 0

1. PREPARATION

1-1 Overview

This Document describes how to connect the ground cable of Patient Cooling Compressor.

1-2 GND CABLE With SCREW Kit (2289497)

Item	Description	Part Number	Qty.
1.	CABLE ASSY	288584	1
2.	SCREW	N9510ZM	1
3.	NUT	N9501BM	1
4.	WASHER	N9501WM	2
5.	PROTECTOR	U0508AC	1
6.	LABEL	U0022BN	1

2. GROUND CABLE CONNECTION

1. Verify that the patient cooling is set on the floor and the power is OFF.




PATIENT COOLING MODULE
ILLUSTRATION 1

Rev 0

2. GROUND CABLE CONNECTION (CONTINUED)


2. Install edge protector as illustration below.

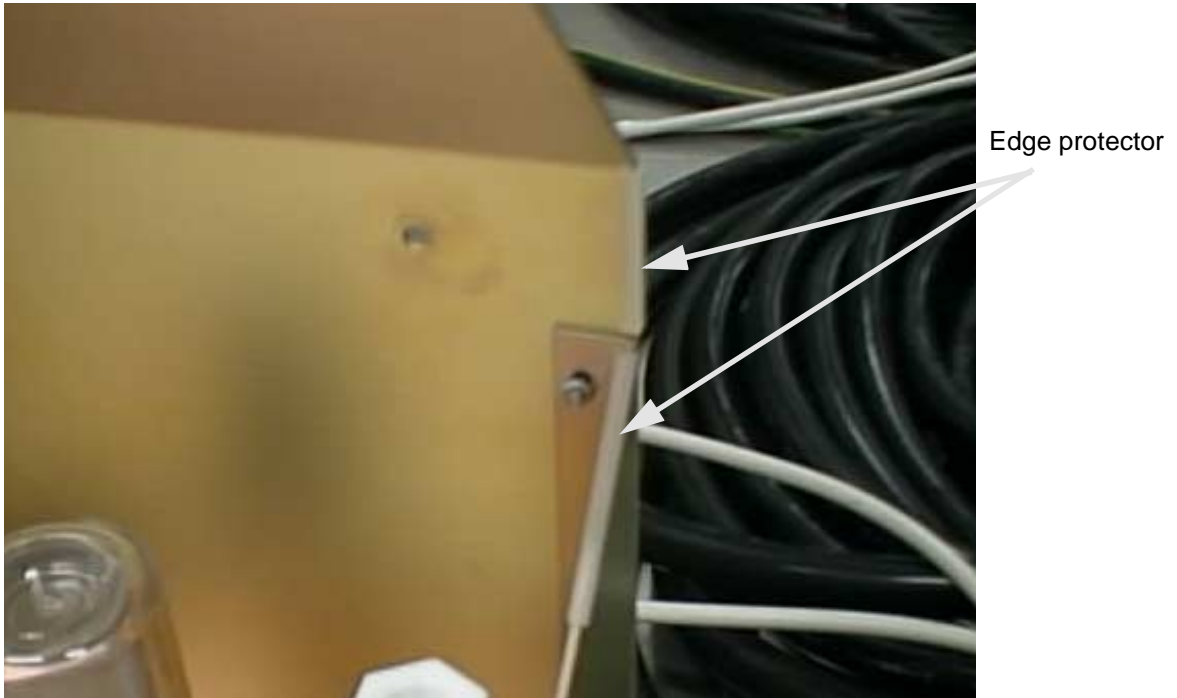

EDGE PROTECTOR
ILLUSTRATION 2

3. Install ground cable to the rear panel.

4. Attach the Ground label.

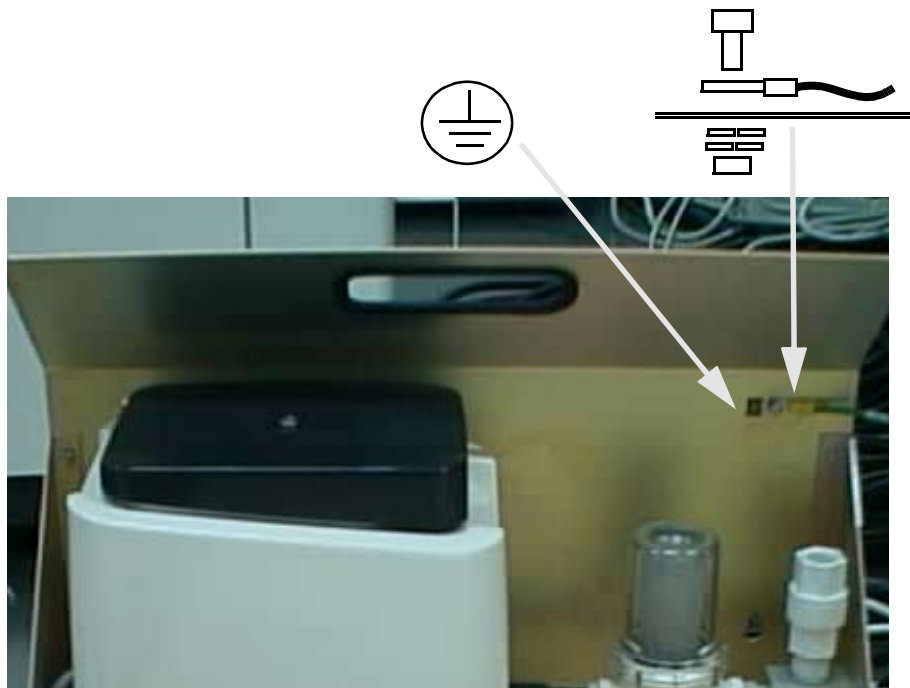

GROUND CABLE
ILLUSTRATION 3

Rev 0

Revision History

Rev	Date	Author	Primary Reasons For Change
0	Mar. 23, 2001	Y. Masumo	Initial Release