

Rev 0

TABLE OF CONTENTS

TABLE OF CONTENTS.....	1
1-Overview	2
2- Procedure	2


Rev 0

1. Overview

This procedure checks the Patient Alert Function.


2. Procedure

1. Squeeze the squeeze bulb and check that beep sound can hear.

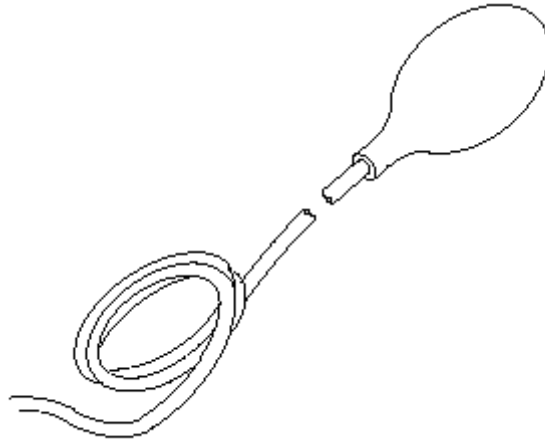

SQUEEZE BULB
ILLUSTRATION 1

2. Push the Reset button of Control box and check that beep sound is stopped.

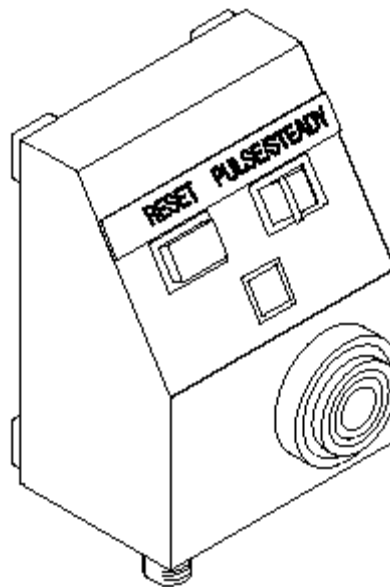

CONTROL BOX
ILLUSTRATION 2

Rev 0

Revision History

Rev	Date	Author	Primary Reasons For Change
0	Jan 26, 2001	Y. Masumo	Initial Version