

# TABLE OF CONTENTS

TABLE OF CONTENTS.....	1
1. Overview .....	2
2. Body Coil Registration .....	2

Rev 0

### 1. Overview

This procedure describes how to remove Body Coil in Config File.

It is necessary to remove Body Coil Config File after RF Output check, Body Power Check, and LV Shim.


### 2. Remove Body Coil

- 1. Click [Service Desktop]..



**SERVICE DESKTOP ILLUSTRATION 1**

- 2. Select "Config File Manager" and Click [Start....]..

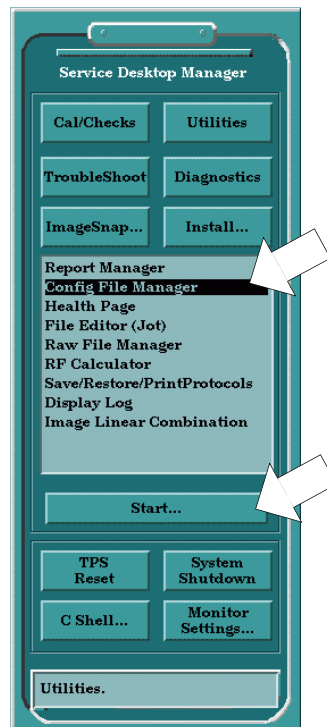


**SERVICE DESKTOP MANGER ILLUSTRATION 2**

Rev 0

## 2. Body Coil Registration (continued)

3. Click [Coil Config File]. .

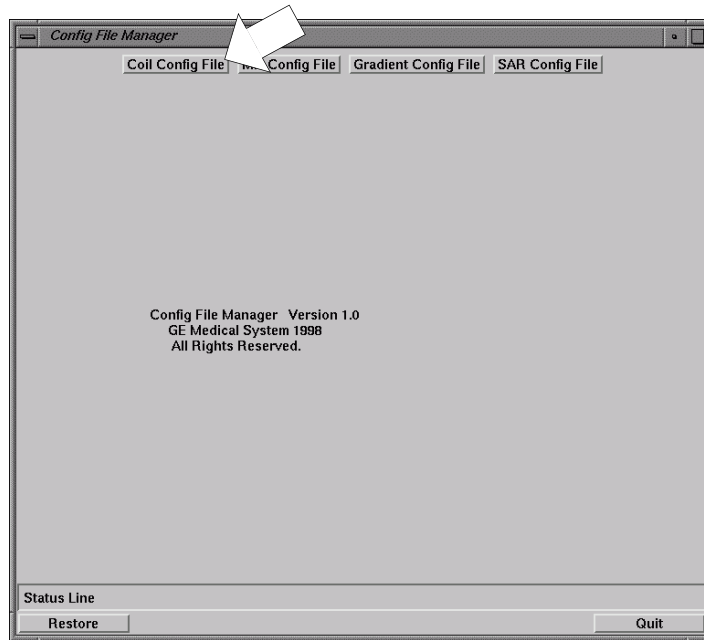


**CONFIG FILE MANAGER**  
ILLUSTRATION 3

4. Select the “**Body**” from “Site Coils” and click [Remove] button.

5. Verify that the Coil is removed from Site Coil.

6. Click [Quit].

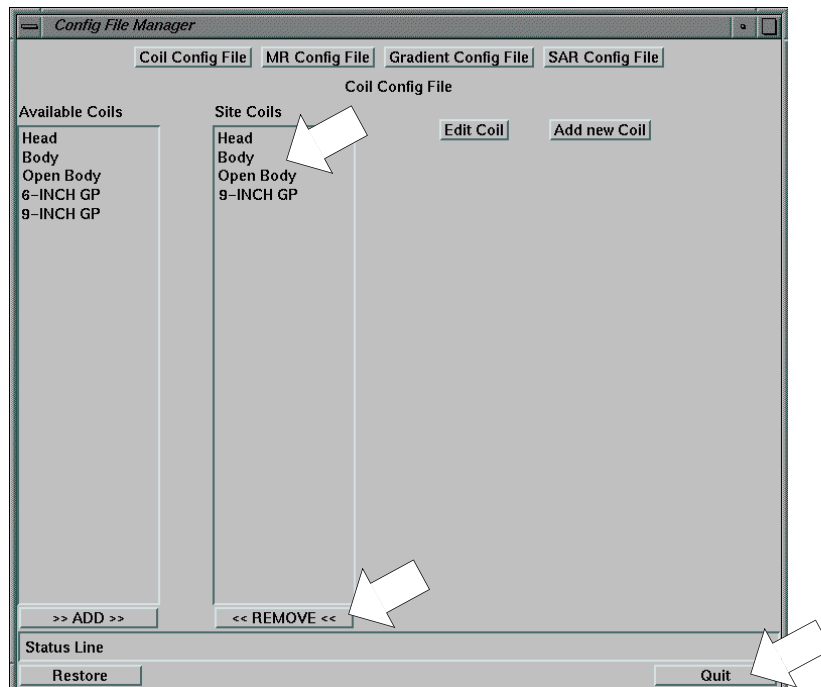

**CONFIG FILE MANAGER**  
ILLUSTRATION 4

Rev 0


**2. Body Coil Registration (continued)**

7. Click [Yes].



**REALLY QUIT**  
ILLUSTRATION 5

8. Click [Save].

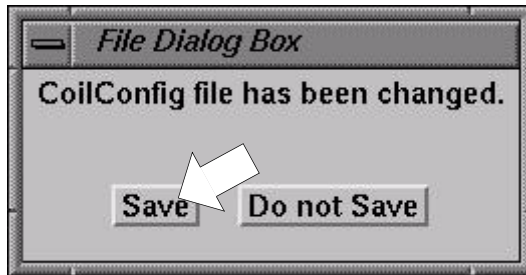

**FILE DIALOG BOX**  
ILLUSTRATION 6

9. It is necessary to reboot the system to update the coil config file.

Rev 0

**Revision History**

<b>Rev</b>	<b>Date</b>	<b>Auther</b>	<b>Primary Reasons For Change</b>
0	Feb 9, 2001	Y. Masumo	Initial Release