

TABLE EMERGENCY RELEASE CHECK

TABLE OF CONTENTS

Table of Contents	1
1.Overview	2
2.Procedure	2


Rev 0

1. Overview

This document describes how to check the Table Emergency Release Function.

2. Procedure

1. Move the Cradle IN.
2. Turn the Cradle release lever and pull the cradle out.
Verify that the cradle is released without any fail..

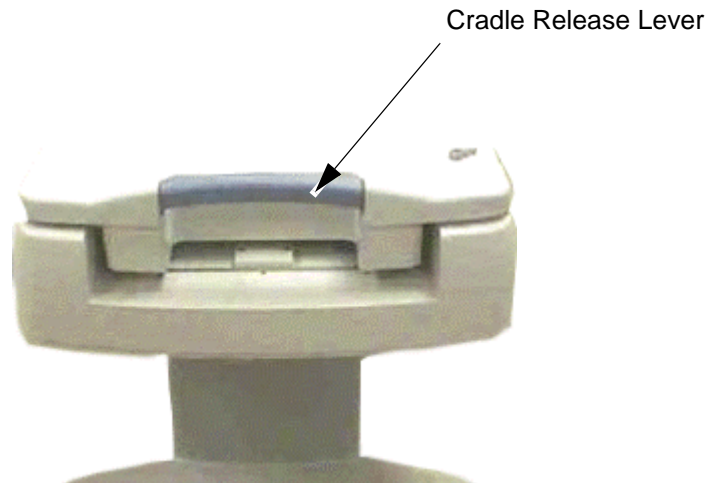

CRADLE RELEASE CHECK
ILLUSTRATION 1

Rev 0

Revision History

Rev	Date	Author	Primary Reasons For Change
0	May 12, 2003	Y. Masumo	Initial Version