

Rev 0

TABLE OF CONTENTS

TABLE OF CONTENTS.....	1
1- Overview	2
2- Procedure	2

Rev 0

1. Overview


MFO2 Mini Viewer is different from MFO1 Mini Viewer.

However, it is necessary to use MFO1 Mini Viewer when measuring the certain coil SNR.

When it is necessary to use MFO1 Mini Viewer, perform the following procedure.

2. Procedure

1. Select the image from browser.
2. Open Command Window by holding the right mouse button and selecting "Service Tool/Command Window".

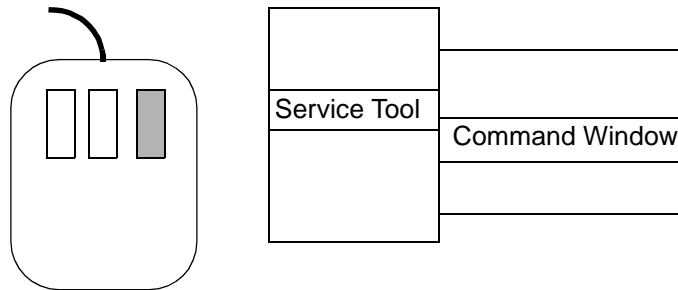

COMMAND WINDOW
ILLUSTRATION 1

3. Enter following command in the command window.
`viewer.sdc -w [Return]`
4. Verify that image is displayed.

Rev 0

Revision History

Rev	Date	Author	Primary Reasons For Change
0	Jan. 28, 2002	Y. Masumo	Initial Release