

Run	GE Part Number	Source Subsystem	Source Assembly	Source Connection	Destination Subsystem	Destination Assembly	Destination Connection
030M	46-271364G517	System Cabinet	Power Panel	J1	Power Cabinet	Power Distribution Unit	28,29,30,N
037T	46-255947G607	System Cabinet	Power Panel	GND	Power Cabinet	Power Distribution Unit	GRN/YEL BUS
040	2103427	Magnet	Magnet	GND			GND Stud
044	46-255947G9	Penetration Panel	Penetration Panel	GND	Power Cabinet	System Support Module	GND
047	46-307832P4	Operator Workspace	Power	J1	Power Cabinet	Power Distribution Unit	31,34,N
048	46-255947G11	Operator Workspace	Power	GND	Power Cabinet	Power Distribution Unit	GRN/YEL
049	46-307829P1	Operator Workspace	Power	J2	Operator Workspace	Power Module	PWR
229	46-301125G98	System Cabinet	System Control Cabinet	J1	Power Cabinet	System Support Module	J3
231	46-301125G100	System Cabinet	System Control Cabinet	J2	Penetration Panel	Penetration Panel	J8
296	46-271692G601	Penetration Panel	Penetration Panel	J18	Main Disconnect Panel	Main Disconnect Panel	
297	46-271693G1	Emergency Cutoff Switch	Emergency Cutoff Switch in		Penetration Panel	Penetration Panel	J18
414	46-317157G3			MIC	Magnet Enclosure	Patient Communication Box	J9
487	46-301125G764	Power Cabinet	System Support Module	J20	Penetration Panel	Multi-coil Switching Filter	J78
488	46-301125G765	System Cabinet	System Control Cabinet	J8	Penetration Panel	Penetration Panel	J80
489	46-301125G766	System Cabinet	System Control Cabinet	J9	Penetration Panel	Penetration Panel	J81
490	46-301125G767	System Cabinet	System Control Cabinet	J10	Penetration Panel	Penetration Panel	J82
701	46-301125G807	System Cabinet	System Control Cabinet	J15			RF Door Switch
702	46-258431G810	System Cabinet	System Control Cabinet	J18	Power Cabinet	System Support Module	J5
703	46-258431G809	System Cabinet	System Control Cabinet	J23	Power Cabinet	Power Distribution Unit	J2
706	46-258431G808	System Cabinet	System Control Cabinet	J14	Power Cabinet	Power Distribution Unit	J3
708	46-317076G602	System Cabinet	System Control Cabinet	J16	Power Cabinet	System Support Module	J27
710	46-317085G603	Power Cabinet	Gradient Driver	J2	System Cabinet	System Control Cabinet	J17
711	46-328079G9	Penetration Panel	F/O Repeater Panel	J91	Magnet Enclosure	Scan Room Interface (SRI)	J11
711	46-328079G9	Penetration Panel	F/O Repeater Panel	J91	Magnet Enclosure	Scan Room Interface (SRI)	J13
711	46-328079G9	Penetration Panel	F/O Repeater Panel	J91	Magnet Enclosure	Scan Room Interface (SRI)	J12
711M/712M	46-328079G9	Penetration Panel	F/O Repeater Board	J1	Penetration Panel	F/O Repeater Panel	J91
711M/712M	46-328079G9	Penetration Panel	F/O Repeater Board	J2	Penetration Panel	F/O Repeater Panel	J91
711M/712M	46-328079G9	Penetration Panel	F/O Repeater Board	J3	Penetration Panel	F/O Repeater Panel	J91
711M/712M	46-328079G9	Penetration Panel	F/O Repeater Board	J5	Penetration Panel	F/O Repeater Panel	J91
711M/712M	46-328079G9	Penetration Panel	F/O Repeater Board	J4	Penetration Panel	F/O Repeater Panel	J91
711V/712V	46-328078G5	System Cabinet	System Control Cabinet	J13	Penetration Panel	F/O Repeater Board	J7
711V/712V	46-328078G5	System Cabinet	System Control Cabinet	J13	Penetration Panel	F/O Repeater Board	J8
711V/712V	46-328078G5	System Cabinet	System Control Cabinet	J13	Penetration Panel	F/O Repeater Board	J9
711V/712V	46-328078G5	System Cabinet	System Control Cabinet	J13	Penetration Panel	F/O Repeater Board	J10
711V/712V	46-328078G5	System Cabinet	System Control Cabinet	J13	Penetration Panel	F/O Repeater Board	J11
712	46-328079G9	Penetration Panel	F/O Repeater Panel	J91	Magnet Enclosure	Physiological Acquisition C	J1
726	46-301125G831	Penetration Panel	Multi-coil Bias Filter (G2)	J77	Power Cabinet	System Support Module	J18
745	46-287688G5	Power Cabinet	Radio Frequency Interface	J4			Run 935
746	2263075	Penetration Panel	Penetration Panel	J44			Run 912

Run	GE Part Number	Source Subsystem	Source Assembly	Source Connection	Destination Subsystem	Destination Assembly	Destination Connection
768TR	46-328000G902	Penetration Panel	Penetration Panel	J20	Power Cabinet	System Support Module	J31
769	46-328000G903	Penetration Panel	Penetration Panel	J15	Power Cabinet	System Support Module	J34
770	46-328000G904	Penetration Panel	Penetration Panel	J11	Power Cabinet	System Support Module	J41
771	46-328000G905	Penetration Panel	Penetration Panel	J47	Power Cabinet	System Support Module	J36
772	46-328000G906	Penetration Panel	Penetration Panel	J14	Power Cabinet	System Support Module	J40
773	46-328000G907	Penetration Panel	F/O Repeater Panel	J1	Power Cabinet	System Support Module	J35
774M	46-258431G908	System Cabinet	System Control Cabinet	J25	Power Cabinet	System Support Module	J32
775	46-328000G915	Power Cabinet	System Support Module	J42	Penetration Panel	Dynamic Disable and Induc	J92
777	46-328000G913	Power Cabinet	System Support Module	J43	Penetration Panel	Dynamic Disable and Induc	J94
788	46-328000G937	Penetration Panel	Penetration Panel	J12	Operator Workspace	Workspace Interface Modul	J7
789	46-328000G938	System Cabinet	System Control Cabinet	J26	Operator Workspace	Workspace Interface Modul	J18
791	46-328000G939	System Cabinet	System Control Cabinet	J30	Operator Workspace	SCSI Serial Expansion	Port 1
805	2114561-38	Operator Workspace	Workspace Interface Modul	J6	Operator Workspace	Keyboard Assembly	
807	2163386	Operator Workspace	SCSI Serial Expansion	SCSI 2?	Operator Workspace	DASM	SCSI 1?
810	2141980-2	Operator Workspace	Workspace Interface Modul	J8	Operator Workspace	SGI Octane Computer	KYBD, MS
811	2153396-3	Operator Workspace	Local Area Network HUB	PORT 2	Operator Workspace	SGI Octane Computer	ETHERNET
812	2188746	Operator Workspace	SGI Octane Computer	Serial Port 1 (RS232)	Operator Workspace	Modem	DB-9
813	2188747	Operator Workspace	Personal Computer	COM 2	Operator Workspace	SGI Octane Computer	Serial Port 2 (RS232)
814	2196990	Operator Workspace	Personal Computer	Keyboard, Mouse, CO	Operator Workspace	Workspace Interface Modul	J9
815	2198983	Operator Workspace	SGI Octane Computer	SCSI on PCI	Operator Workspace	SCSI Serial Expansion	
817	46-328000G958	Operator Workspace	Power	J3	Operator Workspace	Workspace Interface Modul	J17
818	2198993	System Cabinet	System Control Cabinet	J31	Operator Workspace	SCSI Serial Expansion	Ports 6, 7, 8
819	2198995	Operator Workspace	SCSI Expansion Box	IN	Operator Workspace	SGI Octane Computer	SCSI
820	2150533	Operator Workspace	SGI Octane Computer	Monitor port	Operator Workspace	Host Monitor (Black and W	
821	2150533-2	Operator Workspace	SGI Octane Computer	Monitor port	Operator Workspace	Host Monitor (Color)	
836	2241052	Operator Workspace	SGI Octane Computer	BIT3	System Cabinet	System Control Cabinet	J21
837	2263200	Operator Workspace	Serial Converter	J4	Penetration Panel	TCU	J2
838	2263200-2	Operator Workspace	Serial Converter	J3	Penetration Panel	Penetration Panel	J51
841	2263200-3	Penetration Panel	Penetration Panel	J51	Patient Table	Patient Table	J5
842	2263200-4	Penetration Panel	Penetration Panel	J15	Magnet Enclosure	SRI Splitter	J7
843	2260535	Penetration Panel	Penetration Panel	J20	Magnet Enclosure	Magnet Interface Board	J6
849	2263200-5	Magnet Enclosure	Scan Room Interface (SRI)	J4	Magnet Enclosure	SRI Splitter	J1
850	2263200-6	Magnet Enclosure	Scan Room Interface (SRI)	J7	Magnet Enclosure	SRI Splitter	J3
858	2263200-9	Magnet Enclosure	SRI Splitter	J4	Patient Table	Patient Table	J15
859	2263200-10	Magnet Enclosure	Scan Room Interface (SRI)	J3	Patient Table	Patient Table	J14
860	2263200-11	Magnet Enclosure	Scan Room Interface (SRI)	J2	Patient Table	Patient Table	J13
861	2221947	Magnet Enclosure	Operator Display	J4	Magnet Enclosure	Scan Room Interface (SRI)	J5
861	2221947	Magnet Enclosure	Operator Display	J4	Magnet Enclosure	SRI Splitter	J10
861	2221947	Magnet Enclosure	Operator Display	J4	Magnet Enclosure	SRI Splitter	J11

Run	GE Part Number	Source Subsystem	Source Assembly	Source Connection	Destination Subsystem	Destination Assembly	Destination Connection
861	2221947	Magnet Enclosure	Operator Display	J4	Magnet Enclosure	Scan Room Interface (SRI)	J9
861	2221947	Magnet Enclosure	Operator Display	J4	Magnet Enclosure	Scan Room Interface (SRI)	J6
862	2221946	Magnet Enclosure	Operator Display	J1	Magnet Enclosure	Operator Control (Right)	J1
863	2221946	Magnet Enclosure	Operator Display	J2	Magnet Enclosure	Operator Control (Left)	J2
864	2261328	Magnet Enclosure	SRI Splitter	J6	Magnet Enclosure	Bore Light (Left)	
865	2261328	Magnet Enclosure	SRI Splitter	J5	Magnet Enclosure	Bore Light (Right)	
866	2263200-12	Patient Table	Patient Table	J12	Magnet Enclosure	Magnet Interface Board	J3
869	46-317252G4 (ch	Magnet Enclosure	Patient Comfort Module Sp	N/A	Magnet Enclosure	Patient Communication Box	J8
870	46-317157G6	Magnet Enclosure	Patient Comfort Module Mic	N/A	Magnet Enclosure	Patient Communication Box	J7
871	2261329			Laser Diodes	Magnet Enclosure	Magnet Interface Board	J8
873	2263200-13	Magnet Enclosure	Patient Communication Box	J6	Magnet Enclosure	Magnet Interface Board	J1
875	46-317220P19	Patient Table	Patient Table	J1	Patient Table	Coil I/F Electronics	J6
876	46-317220P20	Patient Table	Patient Table	J2	Patient Table	Coil I/F Electronics	J7
877	46-317220P21	Patient Table	Patient Table	J3	Patient Table	Coil I/F Electronics	J8
878	46-317220P22	Patient Table	Patient Table	J4	Patient Table	Coil I/F Electronics	J5
879	2262695	Patient Table	Patient Table	J5	Patient Table	Coil I/F Electronics	J3
880	2262694-2	Patient Table	Patient Table	J9	Patient Table	Coil I/F Electronics	J9
881	2262696	Patient Table	Patient Table	J15	Patient Table	Coil I/F Electronics	J2
882		Patient Table	Patient Table	J14	Patient Table	Limit Switches and Encoder	J4
883		Patient Table	Patient Table	J13	Patient Table	Limit Switches and Encoder	J6
884		Patient Table	Patient Table	J11	Patient Table	Limit Switches and Encoder	J1
885		Patient Table	Patient Table	J12	Patient Table	Limit Switches and Encoder	J5
886		Patient Table	Patient Table	J8	Patient Table	Longitudal Motor Drive	N/A
887	2263076	Power Cabinet	Right/Left Switch	to Run 932	Penetration Panel	Penetration Panel	J4
888	2263076-2	Power Cabinet	Right/Left Switch	to Run 933	Penetration Panel	Penetration Panel	J83
889	2263076	Patient Table	Patient Table	J6	Penetration Panel	Penetration Panel	J4
890	2263076-2	Patient Table	Patient Table	J7	Penetration Panel	Penetration Panel	J83
891	46-320888G3	Patient Table	Patient Table	J6	Patient Table	Coil I/F Electronics	J1
892	46-320888G4	Patient Table	Patient Table	J7	Patient Table	Coil I/F Electronics	J10
893	46-282352G2	Magnet Enclosure	Body Quadrature Hybrid As	J2	Magnet Enclosure	Body Coil Preamplifier	IN
894	46-317758P18	Magnet Enclosure	Physiological Acquisition C	Respiration			Respiration
895	2219097	Magnet Enclosure	Physiological Acquisition C	Peripheral Gating			Peripheral Gating
896	2219096	Magnet Enclosure	Physiological Acquisition C	ECG			ECG
897	2221948	Magnet Enclosure	SRI Splitter	J9	Magnet Enclosure	Operator Display	J3
898	46-317903	Magnet Enclosure	Physiological Acquisition C	Photo Plethysmograph			Photo Plethysmograph
905	2262694	Patient Table	Patient Table	J10	Patient Table	Coil I/F Electronics	J4
906	2263200-14	Patient Table	Patient Table	J11	Penetration Panel	Penetration Panel	J47
910	2263200-15	Operator Workspace	SCSI Serial Expansion	PORT 3	Operator Workspace	Serial Converter	J2
911	2263200-16	Operator Workspace	SCSI Serial Expansion	PORT 5	Operator Workspace	Serial Converter	J1

Run	GE Part Number	Source Subsystem	Source Assembly	Source Connection	Destination Subsystem	Destination Assembly	Destination Connection
912	46-287688G9	Magnet Enclosure	Body Quadrature Hybrid As	J1			run 746
920	2263200-17	Magnet Enclosure	Magnet Interface Board	J2	Penetration Panel	Penetration Panel	J12
921	2263200-18	Patient Table	Patient Table	J8	Penetration Panel	Penetration Panel	J11
926	2263200-23	Penetration Panel	Multi-coil Switching Filter	J78	Patient Table	Patient Table	J9
927	2263200-24	Penetration Panel	Multi-coil Bias Filter (G2)	J77	Patient Table	Patient Table	J10
929	2263200-26	System Cabinet	System Control Cabinet	J5	Penetration Panel	Penetration Panel	J7
930	2259428	Power Cabinet	System Support Module	J37	Patient Comfort Compresso	Patient Comfort Compresso	
931	2259428-2	Power Cabinet	System Support Module	J47	Patient Comfort Compresso	Patient Comfort Compresso	
932	46-287688G7	Power Cabinet	Right/Left Switch	NC			Run 887
933	46-287688G8	Power Cabinet	Right/Left Switch	NO			Run 888
934	2263200-27	Penetration Panel	Penetration Panel	J8	Magnet Enclosure	Body Coil Preamplifier	Output
935	2263075	Power Cabinet	Radio Frequency Interface	join to Run 745	Penetration Panel	Penetration Panel	J44
943	2275301			Table Pivot Sensor			Run 944
944	2263200-34	Magnet Enclosure	Magnet Interface Board	J4			Run 943
1001	2283335	Power Cabinet	Gradient Driver	X-GCA	Penetration Panel	Gradient Filter Box	1,2,GND
1002	2283336	Power Cabinet	Gradient Driver	Y-GCA	Penetration Panel	Gradient Filter Box	3,4,GND
1003	2283337	Power Cabinet	Gradient Driver	Z-GCA	Penetration Panel	Gradient Filter Box	5,6,GND
1004	2283338	Penetration Panel	Gradient Filter Box	1,2,GND	Magnet Enclosure	Gradient Coil	1,3
1005	2283339	Penetration Panel	Gradient Filter Box	3,4,GND	Magnet Enclosure	Gradient Coil	5,7
1006	2283340	Penetration Panel	Gradient Filter Box	5,6,GND	Magnet Enclosure	Gradient Coil	9,11
1007	2279862	Power Cabinet	Power Distribution Unit	RF Coil Cooling PLUG	Cooling Cabinet	Cooling Cabinet	TM1 (1,2)
1008	2279864	Power Cabinet	Power Distribution Unit	GND	Cooling Cabinet	Cooling Cabinet	GND
1009	2280055	Power Cabinet	Power Distribution Unit	33,36,GND	Penetration Panel	TCU	J1
1010	2280056	Magnet	Magnet	GND	Magnet Enclosure	Scan Room Interface (SRI)	GND
1011	2282839	Penetration Panel	TCU	J3	Magnet	Magnet	J13
1012	2282838	Penetration Panel	TCU	J4	Magnet	Magnet	J14
1013	2282840	Penetration Panel	TCU	J5	Magnet	Magnet	J15
1014	2284671	Penetration Panel	Dynamic Disable and Induc	J72	Magnet Enclosure	Output Bias Box	DC in 1
1015	2284672	Penetration Panel	Dynamic Disable and Induc	J75	Magnet Enclosure	Output Bias Box	DC in 2
1016	2279863	Penetration Panel	Penetration Panel	J14	Magnet Enclosure	Scan Room Interface (SRI)	J1
1017	2280058	Magnet Enclosure	SRI Splitter	J8	Magnet Enclosure	Magnet Interface Board	J7
1018	2280057	Magnet Enclosure	Magnet Interface Board	J5	Magnet Enclosure	Pinch Switch	A1, A2, A3
1019	2284412	Patient Table	Patient Table	J4	Penetration Panel	MA Module	J1
1020	2284413	Patient Table	Patient Table	J1	Penetration Panel	MA Module	J2
1021	2284414	Patient Table	Patient Table	J2	Penetration Panel	MA Module	J3
1022	2284415	Patient Table	Patient Table	J3	Penetration Panel	MA Module	J4
1023	2288234	Magnet Enclosure	Scan Room Interface (SRI)	J8	Magnet Enclosure	SRI Splitter	J2
1024	2288584	Power Cabinet	System Support Module	GND	Patient Comfort Compresso	Patient Comfort Compresso	GND
1025	2294342	Magnet Enclosure	FILTER BOX 2	J1	Penetration Panel	Penetration Panel	J89

Run	GE Part Number	Source Subsystem	Source Assembly	Source Connection	Destination Subsystem	Destination Assembly	Destination Connection
1026	2294343	Magnet Enclosure	FILTER BOX 3	J2	Magnet Enclosure	Operator Display	J5
1027	2284838	Magnet Enclosure	Scan Room Interface (SRI)	GND	Magnet Enclosure	Magnet Interface Board	GND