

# TABLE OF CONTENTS

TABLE OF CONTENTS .....	1
1- Overview .....	2
2- Procedure .....	2

Rev 0

### 1. Overview

It is necessary to adjust gamma according to the following procedure for Signa Ovation3 Systems with PC Host.


### 2. Procedure:

- 1. Click [Service Desktop]..



**SERVICE DESKTOP**  
ILLUSTRATION 1

- 2. Click [C Shell....].

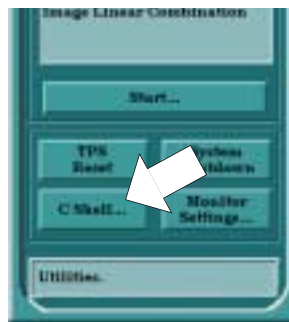

**SERVICE DESKTOP MANGER**  
ILLUSTRATION 2

Rev 0

**2. Procedure (continued):**

3. Verify the current gamma value by typing as follows.

```
{sdc@hanawa}[1] xgamma [Return]
-> Red 1.500, Green 1.500, Blue 1.500
```

4. Type as follows.

```
{sdc@hanawa}[2] cd /usr/pixmaps [Return]
{sdc@hanawa}[3] vi colorPix.cfg [Return]
```

5. Move the cursor to the gamma value.

```
gamma=[ ].[ ]
```

6. Press "X" to delete the value. Ignore the warning sign and hit [Return] key.

7. Hit "A" and to enter the new value.

8. Enter "1.1" as a new vale.

```
gamma=1.1 [Return]
```

9. Hlt [Esc] Key twice.

10. Enter ":wq!" and hit [Return] key. If you need to quit without saving, enter ":q!" hit [Return] key.

```
:wq! [Return]
```

**Revision History**

<b>Rev</b>	<b>Date</b>	<b>Author</b>	<b>Primary Reasons For Change</b>
0	Nov 27, 2002	Y. Masumo	Initial Version